



# EXO AND EXO PRO UPHOLSTERY INSTALLATION INSTRUCTIONS



Please read these instructions carefully before beginning the installation. Failure to understand and follow installation instructions may result in injury to installer and/or end user and may void the warranty. If you have any questions call Sunrise Medical Technical support at 800-333-4000

## 1. Parts included in the Kit:

- a. Back Upholstery

## 2. Tools required


- a. Scissors
- b. Tape measure

### WARNING



Tension adjustable straps should be installed when chair is unfolded. **DO NOT** overtighten straps, as this will cause undue strain upon the chair frame. The difference in width between the top and bottom of the backrest should not exceed +/- 0.5" (+/- 13mm).

## A. INSTALLING ADJUSTABLE UPHOLSTERY

1. Remove any existing upholstery
2. Install top strap through the slot or hook in the push handles first (A). Cut to length. Figures 1 & 2
3. Mount the short side of the backrest straps. Figure 3
4. Next mount the long side of the backrest straps to the opposite back post. Figure 4

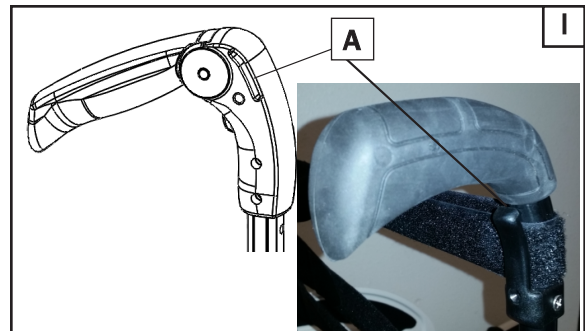


6. Mount the crocodile fixator (B). The fixator should be placed in the middle of the straps. Figures 5 & 6
7. Trim long strap as needed (C) for correct fixator placement. Figure 7
8. Install straps as required for specified height. Strap spacing should not exceed 1-1/2" (38mm).

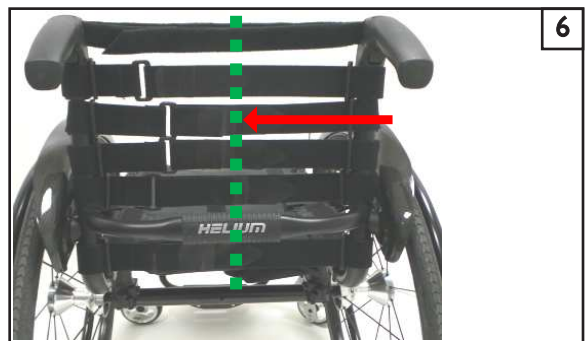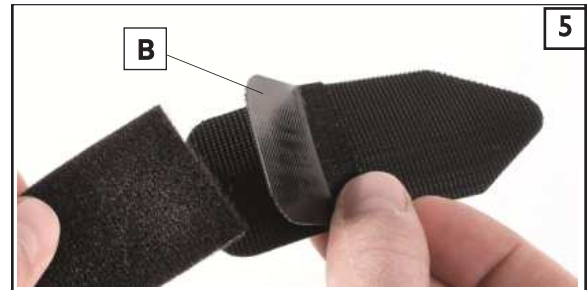


**B. INSTALLING UPHOLSTERY AROUND BRACKETS**

1. Cut opening for Aluminum Side Guard, Carbon Side Guard, Desk Side Guard, or Half Folding Back.
2. Measure and cut slots on both sides (D) to clear side guard bracket or half folding back brackets. Figures 8 & 9

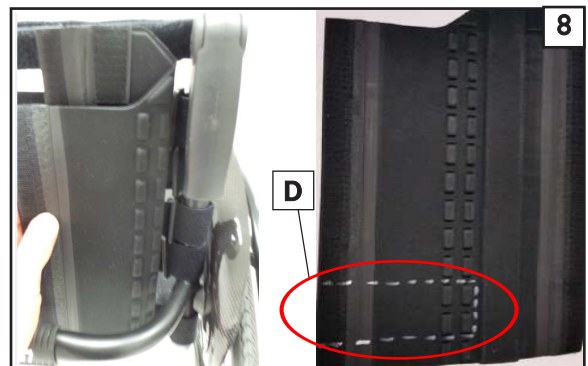


**C. INSTALLING UPHOLSTERY AROUND ACCESSORIES**

1. Cut opening for Height-Adjustable Push Handles, Auto-Fold Stabilizer Bar, or Flip-Up Stabilizer Bar.
2. Measure and cut openings on both sides (E) to go around push handle brackets or stabilizer bar. Figure 10
3. Completed assembly of side covers. Figure 11 (Shown without cut-outs)

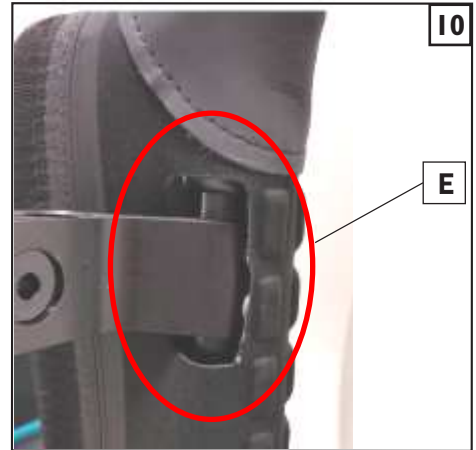


**D. TOP PAD**

1. Mount backrest pad (F). Figure 12

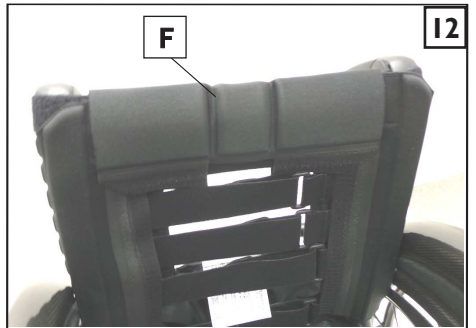

**E. MOUNT COVER**

1. Fold cover over top strap. Adjust as needed to center the cover on the chair. Figures 13 & 14

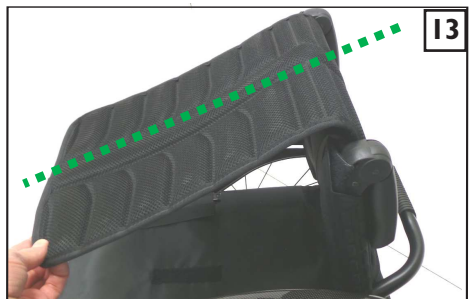

**F. INSPECT**

1. Ensure that cut-outs clear brackets or accessories.
2. Ensure that straps and upholstery are centered on the chair.
3. Ensure that the difference in width between the top and bottom of the backrest does not exceed +/- 0.5" (+/- 13mm).

**DEALER/TECHNICIAN WARNING**** WARNING**

Attention dealers and qualified technicians, do not operate or service this wheelchair without first reading the owners manual. If you do not understand the instructions and warnings in the owners manual please contact the Quickie Technical Service Department before operating and/or servicing the Quickie wheelchair. Failure to do so may result in damage and/or injury.

Sunrise Medical • 2842 N. Business Park Ave. • Fresno, CA 93727 • USA  
In Canada (800) 263-3390