



Please read these instructions carefully before beginning the installation. Failure to understand and follow installation instructions may result in injury to installer and/or end user and may void the warranty. If you have any questions call Sunrise Medical Technical support at 800-333-4000.

Kit parts:

1. 1 ea Pad
2. 1 ea Pad bracket assm
3. 1 ea Spacer bracket

4. 1 ea Offset bracket 1"
5. 2 ea 1/2" Screws
6. 2 ea 1" Screws
7. 2 ea Washers

Tools needed for assembly:

1. 5/32 hex key
2. Scissors

A. PREP THE PAD: **1**

- a. Determine which mounting holes will be used on the pad.
- b. Using the scissors, poke holes through the pad cover to expose the threads of the mounting holes. Figure 1

B. ASSEMBLING THE PAD: **2**

- a. Assemble screws (6) and washers (7) through mounting brackets (8)(3).
- Note:* Use screws (5) when not using spacer (3).
- b. Using the 5/32 hex key, mount the assembly to the pad (1).
- c. Tighten securely. Figure 2

C. OFFSET BRACKET: **3**

- a. Offset bracket (4) can be used in place of pad bracket piece Item 8. The offset bracket (4) can be used with or without Item 9, Figure 3.

D. OPPOSITE SIDE: **4**

- a. Assembly shown is for the right side. Assemble pad in opposite direction for the left side, Figure 4.



WARNING

DEALER/TECHNICIAN WARNING

Attention dealers and qualified technicians, do not operate or service this device without first reading the owners manual. If you do not understand the instructions and warnings in the owners manual please contact the Quickie Technical Service Department before operating and/or servicing the Quickie device. Failure to do so may result in damage and/or injury.

For more information and important warnings, see your chair's owner's manual at www.Sunrisemedical.com/Help-support

