



Please read these instructions carefully before beginning the installation. Failure to understand and follow installation instructions may result in injury to installer and/or end user and may void the warranty. If you have any questions call Sunrise Medical Technical support at 800-333-4000.

ALR Knuckle Cover parts:


- 1 ea ALR Knuckle Cover

Tools needed for assembly:

1. none

A. ALR Knuckle Cover Installation:

1. Open up the ALR Knuckle Cover by unzipping the zipper and detaching the Velcro.
2. Place the Hanger/Frame Interface on the ALR through hole A on the knuckle cover. (Fig. 1)
3. Place the lever on the ALR through hole B on the knuckle cover. (Fig. 1)
4. Adjust the cover so that it fits snugly over the top of the ALR knuckle and extension tube adjustment bars. (Fig. 2)
5. Secure the Velcro and close the zipper. (Fig. 3 & 4)

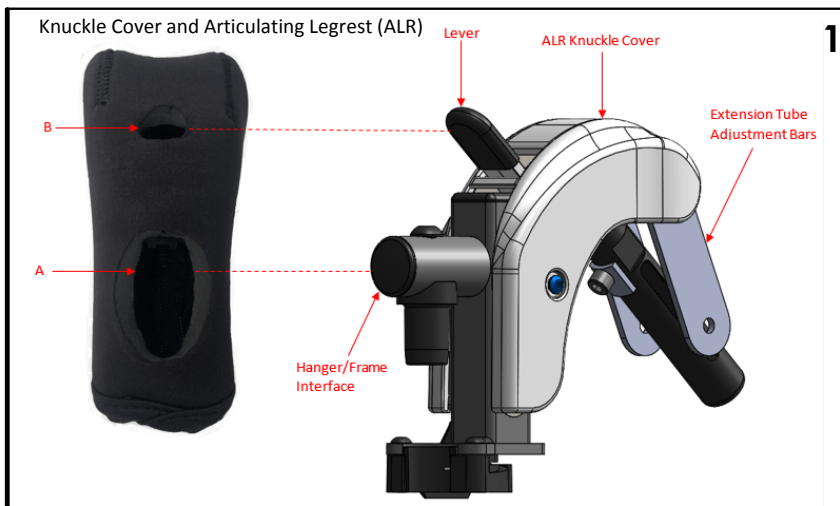

DEALER/TECHNICIAN WARNING



WARNING

Attention dealers and qualified technicians, do not operate or service this device without first reading the owners manual. If you do not understand the instructions and warnings in the owners manual please contact the Quickie Technical Service Department before operating and/or servicing the Quickie device. Failure to do so may result in damage and/or injury. **Find more information and important warnings see the wheelchair owner's manual or at: www.SunriseMedical.com.**

**Sunrise Medical 2842 Business Park Ave Fresno, CA 93727 USA
In Canada (800) 263-3390**