



## Q700-UP M SILENCER KIT INSTRUCTION SHEET



Please read these instructions carefully before beginning the assembly. Failure to understand and follow assembly instructions may result in injury to technician and/or end user and may void the warranty. If you have any questions call Sunrise Medical Technical Support at +1 (800) 333-4000.

### Parts included in kit:

1. Washer DIN988 14x20x1 A2 (qty 2)
2. Washer DIN988 25x35x1 ST (qty 2)
3. O Ring ISO3601 6,75x1.78 NBR (qty 2)

### Tools required

- |                       |                       |
|-----------------------|-----------------------|
| 1. 2 x 4 mm Allen key | 4. 16mm Socket Wrench |
| 2. 6mm Allen key      | 5. 18mm Socket Wrench |
| 3. 13mm Spanner       | 6. Torque Wrench      |



1. Lift the wheelchair so that the drive wheels can be detached. Use the 16mm socket wrench to loosen the nuts (Fig 1).

2. With the 18mm socket wrench, loosen the two M12 bolts that fix the front tie down bracket and remove the bracket (Fig 2). Keep all hardware.

3. a. Using two 4mm Allen keys, loosen one of the M5 bolts that fix the sliding bolt of the lower front parallel arm (Fig 3).

b. Push out the sliding bolt and detach the lower parallel arm (Fig 4).

c. Detach the lower, large sliding bolt (Fig 5).

Note: Keep all hardware and sliding bolts from steps 3A, 3B, and 3C.



4. The 25x35x1 washer needs to be placed on the inside of the parallel arm over the large sliding bolt (Fig 6).

All hardware used in this step was removed in Step 3 and will be used for re-assembly (in reverse order).

a. Attach the parallel arm over the sliding bolt (Fig 7).

b. Align the front part with the steering head and fit the small sliding bolt (Fig 8).

c. Secure it with the M5 bolt. Torque to 5 ft-lbs (7 Nm).

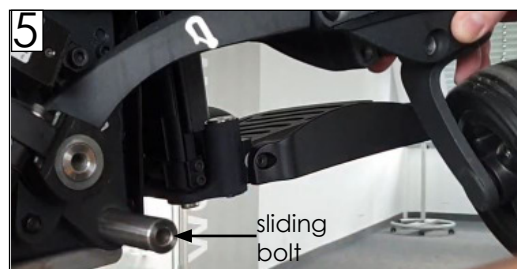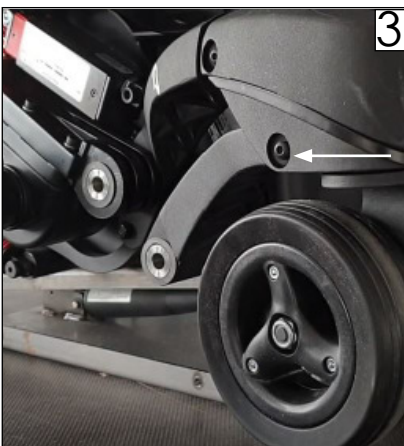

Sunrise Medical 2842 N. Business Park Ave Fresno, CA 93727 USA


Customer service: +1(800) 333-4000  
or visit [www.SunriseMedical.com](http://www.SunriseMedical.com)




© 2020 Sunrise Medical (US) LLC  
08/2020 253003 Rev. A Page 1 of 2



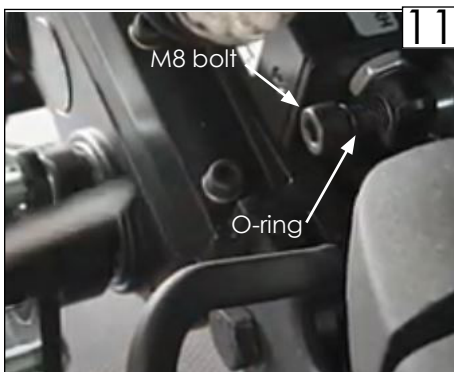

Please read these instructions carefully before beginning the assembly. Failure to understand and follow assembly instructions may result in injury to technician and/or end user and may void the warranty. If you have any questions call Sunrise Medical Technical Support at +1 (800) 333-4000.



5. Add the 14x20x1 washer over the front M12 bolt after you have fitted it through the tie down bracket (Fig 9). Attach M12 bolts (that were removed in step 2) through the sliding bolts. Torque to 50 ft-lbs (68 Nm). 
6. As a next step slide one of the O-Rings over the bottom Caster Lock fixation bolt.
  - a. First, using the 13mm Spanner, loosen the counter nut on the inside and take it off (Fig 10).
  - b. Next, using the 6mm Allen key, loosen and take off the M8 bolt (Fig 10).
  - c. Attach the O-Ring and fit the bolt back to its place (Fig 11).
  - d. Attach the counter nut and tighten it. Torque to 10 ft-lbs (13 Nm). 
7. Attach the drive wheel. Torque lug nuts to 25 ft-lbs (34 Nm). 
8. Complete the update on the opposite chassis side following the same procedure.

**⚠️ DEALER/TECHNICIAN WARNING**

Attention dealers and qualified technicians, do not operate or service this device without first reading the owners manual. If you do not understand the instructions and warnings in the owners manual please contact the Sunrise Medical Technical Service Department before operating and/or servicing the Quickie device. Failure to do so may result in damage and/or injury. Find more information and important warnings see the wheelchair owner's manual or at: [www.Sunrisemedical.com](http://www.Sunrisemedical.com).

**Sunrise Medical 2842 N. Business Park Ave Fresno, CA 93727 USA**