



ALLCOURT / MATCH POINT 5TH WHEEL SUPPORT




Please read these instructions carefully before beginning the assembly. Failure to understand and follow assembly instructions may result in injury to technician and/or end user and may void the warranty. If you have any questions call Sunrise Medical Technical Support at +1 (800) 333-4000.

ITEM NO.	QTY	TITANIUM ASSEMBLY	ALUMINUM ASSEMBLY
1	5	3/4" SADDLE THIN	3/4" SADDLE THIN
2	2	3/4" INSERT PLUG	3/4" INSERT PLUG
3	1	SEAT CLAMP W/GROOVE	SEAT CLAMP W/GROOVE
4	1	SEAT CLAMP	SEAT CLAMP
5	10	1/4 WASHER	1/4 WASHER
6	1	1/4-28 X 2-1/2 HEX CAP SCREW TI	1/4-28 X 2-1/2 G8 HEX CAP SCREW
7	1	TUBE SUPPORT SEAT REAR TI	TUBE SUPPORT SEAT REAR TI
8	1	1/4-28 X 2 HEX CAP SCREW TI	1/4-28 X 1.85 BUTTONHEAD CAPSCREWS
9	2	ANTI TIP CLAMP	ANTI TIP CLAMP
10	1	1/4-28 X 1-3/4 HEX CAP SCREW TI	1/4-28 X 1-3/4 BUTTONHEAD CAP SCREW
11	2	1/4-28 X 1-3/4 HEX CAP SCREW TI	1/4-28 X 1-1/2 BUTTONHEAD CAP SCREW
12	2	STRAP SUPPORT	STRAP SUPPORT
13	5	1/4-28 NYLOCK NUT	1/4-28 NYLOCK NUT

The following assembly instructions are for the 5th Wheel Support for the RGK AllCourt and Match Point sportchairs.

To connect 5TH Wheel Support:

1. Unscrew nut (item 13a) from bolt (item 6).
2. Connect clamp (items 3 and 4) to rear cross brace on seat frame (detail A). Screw nut back onto bolt (items 13a and 6). Leave loose.
3. Remove nuts, washers, and bolts (items 8, 10 and 13b) from anti-tip clamp (item 9). Install clamps around anti-tip tube at rear of chair and reconnect all hardware (detail A). Leave loose.
4. Slide anti-tip clamp (item 9) along anti-tip tube. When satisfied with positioning tighten all nuts and bolts on both clamps at seat and at anti-tip.
5. If necessary for height adjustment remove nut, bolts, saddles, and washers (item 11) from tube support (item 7) and reattach in next set of adjustment holes.

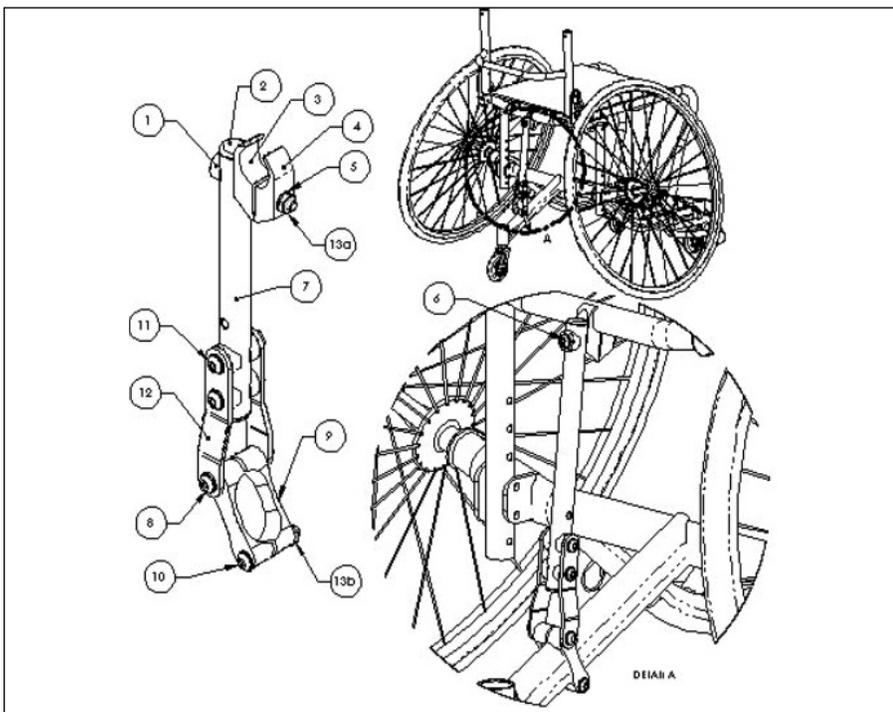

⚠️ DEALER/TECHNICIAN WARNING

Attention dealers and qualified technicians, do not operate or service this device without first reading the owner's manual. If you do not understand the instructions and warnings in the owner's manual please contact the Sunrise Technical Service Department before operating and/or servicing the RGK wheelchair. Failure to do so may result in damage and/or injury.

Find more information and important warnings in the wheelchair owner's manual or at: www.SunriseMedical.com



Customer Service: +1 (800) 333-4000
In Canada (800) 263-3390
or visit www.SunriseMedical.com

© 2023 Sunrise Medical (US) LLC
07/2023 P/N 950158 Rev. B
Pg 1 of 1