

**INSTRUCTION SHEET  
 ZIPPIE TS Z-LOCK  
 ASSEMBLY INSTRUCTIONS**

Patent Pending

The following instructions are for installing the Z-Lock assembly on the New Zippie TS, and the Zippie TS SE with 12" wheels.

ITEM NO.	QTY.	DESCRIPTION
1	2	CLAMP & LOCKING LEVER ASSEMBLY
2	2	CABLE
3	2	LOWER LOCKING ASSEMBLY
4	2	12" WHEEL WITH LOCKING HUB

To install locking hub:

1. Remove wheel from chair, keep quick release axle (Figure 2).
2. Remove 1-3/4" bolt from axle (Figure 2) discard bolt but save nut. Slide hub spacer over hub of 12" wheel axle (Figure 3).
3. Slide large hole of lower locking assembly (Item 3) over same hub so it's butted up against the hub spacer (Figure 3).
4. Using 3" bolt, axle spacer, and nut saved from step 2 attach lower locking assembly to axle through the same hole the 1-3/4" bolt was in. Tighten securely.
5. Remove screw from the clamp assembly (Item 1, Figure 1). Wrap clamps around push handle back tube, reinsert screw, slide to desired position then tighten securely.
6. Insert quick release axle, from old wheel, through new wheel and the 12" wheel axle (Figure 2).
7. Repeat steps on opposite side.

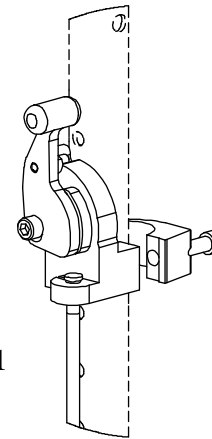


Figure 1

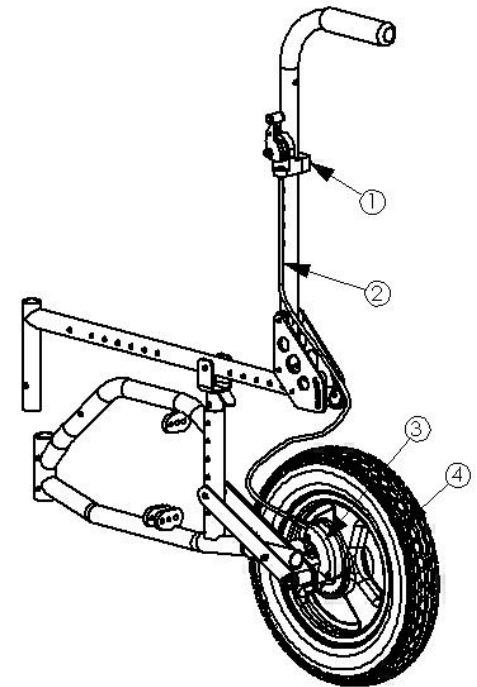


Figure 3


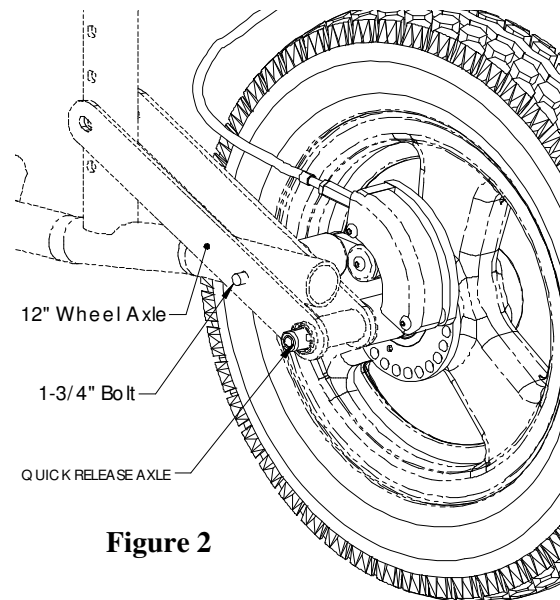


Figure 2

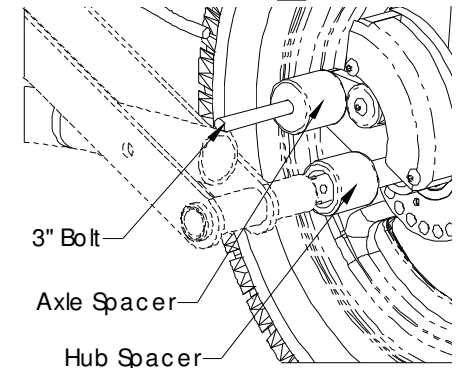

FOR DEALER USE Installed by: \_\_\_\_\_ Date: \_\_\_\_\_ (retain as record of installation)