

## Folding Zippie IRIS Footlock Cable Ordering Guide

Instructions: Select the appropriate right side and left side cable lengths based on; Chair width & Roller to Axle Pos Spacing Depth roller pos & foot pedal pos. options listed on the original chair order.

LEFT FOOT LOCK CABLE GUIDE					
CHAIR WIDTH	ROLLER TO AXLE POS SPACING (DEPTH) <small>Use guide in lower right hand corner to calculate.</small>	LEFT CABLE LENGTH			
		Footlock Left Cable SHORT 342016 (22.5" long)	Footlock Left Cable LONG 342017 (26.5" long)	Footlock Left Cable 406MM 134012 (16" long)	Footlock Left Cable 483MM 342013 (19" long)
10	9-10	-	-	X	-
	11-12	-	-	X	-
	13-14	-	-	-	X
11	9-10	-	-	X	-
	11-12	-	-	X	-
	13-14	-	-	-	X
12	9-10	-	-	X	-
	11-12	-	-	X	-
	13-14	-	-	-	X
13	9-10	-	-	X	-
	11-12	-	-	X	-
	13-14	-	-	-	X
14	9-10	-	-	X	-
	11-12	-	-	X	-
	13-14	-	-	-	X
15	9-10	-	-	X	-
	11-12	-	-	-	X
	13-14	-	-	-	X
16	9-10	-	-	X	-
	11-12	-	-	-	X
	13-14	-	-	-	X
17	9-10	-	-	X	-
	11-12	-	-	X	-
	13-14	-	-	-	X
18	9-10	-	-	X	-
	11-12	-	-	X	-
	13-14	-	-	-	X

RIGHT FOOT LOCK CABLE GUIDE					
CHAIR WIDTH	ROLLER TO AXLE POS SPACING (DEPTH)	RIGHT CABLE LENGTH			
		Footlock Right Cable SHORT 342016 (22.5" long)	Footlock Right Cable LONG 342017 (26.5" long)	Footlock Right Cable 406MM 134012 (16" long)	Footlock Right Cable 483MM 342013 (19" long)
10	9-10	-	-	X	-
	11-12	-	-	X	-
	13-14	-	-	-	X
11	9-10	-	-	X	-
	11-12	-	-	X	-
	13-14	-	-	-	X
12	9-10	-	-	X	-
	11-12	-	-	X	-
	13-14	-	-	-	X
13	9-10	-	-	-	X
	11-12	-	-	-	X
	13-14	X	-	-	-
14	9-10	X	-	-	-
	11-12	X	-	-	-
	13-14	X	-	-	-
15	9-10	X	-	-	-
	11-12	X	-	-	-
	13-14	X	-	-	-
16	9-10	X	-	-	-
	11-12	X	-	-	-
	13-14	X	-	-	-
17	9-10	-	X	-	-
	11-12	-	X	-	-
	13-14	-	X	-	-
18	9-10	-	X	-	-
	11-12	-	X	-	-
	13-14	-	X	-	-

3

i. This depth is calculated by the rear wheel size,

Use the ordering guide to confirm the appropriate combination listed below.

COMBINATIONS:

ASSEMBLY NUMBER:	LEFT CABLE	RIGHT CABLE
243052	134012	134012
243053	134013	134013
243054	134012	134013
243055	134013	342016
243056	134012	342016
243057	134012	342017
243058	134013	342017

REAR WHEEL SIZE	ROLLER POS	FOOT PEDAL POS	DEPTH RANGE (ROLLER TO FOOT PEDAL DISTANCE)	ROLLER TO AXLE POS SPACING (DEPTH)
12-16	1	14	14	9-10
		15	15	
		16	16	11-12
		17	17	
	2	15	15	9-10
		16	16	9-10
		17	17	11-12
		19	19	13-14
	3	15	15	9-10
		16	16	
		17	17	11-12
		19	19	
	4	16	16	9-10
		17	17	
19		19	11-12	
5	17	17	9-10	
	19	19		
20-24	1	17	17	11-12
		18	18	13-14
		19	19	
	2	18	16	11-12
		19	17	13-14
	3	18	15	11-12
		19	16	
	4	19	15	11-12
	5	19	14	9-10