

Replacement Frame Guide for frames built prior to S/N R4-023230

Step 1 - order correct size frame from 1 of the following pages:
STANDARD ADJUSTABLE FRAME after S/N R4-023230 or
ULTRA LIGHT SEAT ADJUSTABLE FRAME after S/N R4-023230 or
ACTIVE RIGID FRAME after S/N R4-023230

Step 5 - order 2 armrest receiver mounts from
the following page:
PADDED SWING-AWAY ARMREST RECEIVER
if so equipped

Step 3 - order correct size seat rail kit from
the following page:
STANDARD SEAT SLING & CUSHION after S/N R4-023230
N/A for Ultra Light Seat Frame and
Active Rigid Frame

Step 2 - order correct size L & R axle plate kit from:
AXLE PLATE page
N/A for Active Rigid Frame
or
REAR SUSPENSION AXLE BRACKET

Step 4 - order correct size footrest from:
ALUMINUM TUBULAR for 17-20" frame width after S/N R4-023230 page
No new foot plate necessary for 12-16 inch frame width

