


REV	ECO	DESCRIPTION	DATE	DESIGNER
A	ECO-0007795	ELEGANCE ELR INSTRUCTION	04/23/14	LANGE

8.50 INCHES



Please read these instructions carefully before beginning the installation. Failure to understand and follow installation instructions may result in injury to installer and/or end user and may void the warranty. If you have any questions call Sunrise Medical Technical Support at 800-333-4000


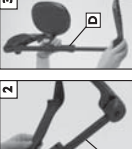

Your kit contains 2 complete Elevating Legrest assemblies.

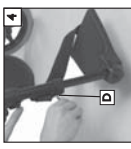
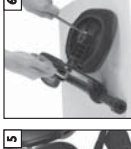

A. FOOTPLATES 1 2 3
Use the seating footplate and extension tube.
 a. Loosen the set screw (A) and do not remove it.
 b. Remove the heel loop (B).
 c. Remove the footplate (C) from the hanger.
 d. Insert the set screw (D) with a 4mm hex key.
Do Not Tighten!
 e. Insert footplate and extension tube into ELR.



B. ADJUSTMENT 4 5 6
Insert the footplate and extension tube into ELR.
 a. Adjust the footplate to the correct height (2" above riding surface) and tighten the screw.
 b. Use the 4mm hex key (F) to adjust the calf pad.
 c. Use the 13mm wrench and 4mm hex key to adjust the calf pad angle, or depth. After adjustment is complete tighten the hardware.

C. ELEVATE YOUR FOOTREST 7 8
Using the ergonomic knob:
 a. Turn the knob to elevate your legrest. You can adjust it when red is visible under the knob.
 b. After adjustment to make sure the ELR is locked, turn the knob until the red is no longer visible. The red is visible under the knob.

⚠️ WARNING
 Avoid the Prone and Inverted techniques. If you have any questions or do not understand the instructions and warnings in this Instruction sheet, please contact the Sunrise Medical Technical Service Dept. before operating and/or servicing the Breezy legrest. Improper use may result in damage and/or injury.

Sunrise Medical • 2942 Business Park Ave. • Fremont, CA 97727 • USA In Canada (800) 263-3190
 Customer Service: 800.333.4000 ©2014 Sunrise Medical (US) LLC
 www.SunriseMedical.com I 14 IS0136 Rev.A

11.0 INCHES

FIRST PAGE OF INSTRUCTION SHEET

MATERIAL NOTES

1. MATERIAL: INK: PERMANENT BLACK NON-SMEARING, PAPER: STOCK FROM PRINTER
2. ADHESIVE: N/A
3. INSPECTION: VID - (VISUAL INSPECTION ONLY)
4. ARTWORK AND CONTENT PROVIDED BY MARKETING
5. PRINTED FRONT BACK

PRODUCTION NOTES

1. COLORS: N/A
2. MULTIPLE PAGE
3. SADDLE STITCHED
4. NO TRIM
5. NO BLEED
6. INSPECTION: MARKETING VID



ALL DIMS ARE IN INCHES UNLESS OTHERWISE STATED
 GMS - LITERATURE CONTROL PROCEDURE (NA.5.3.1.F4)

DESCRIPTION/TITLE

ELEGANCE ELR INSTRUCTION

PART NUMBER

IS00136

Original filename (file extension) and source file:

IS00136_Elegance_ELr_dwg.qxd (Quark express)