

**JAY**<sup>®</sup>**ARMTROUGH INSTALLATION SHEET**

Please read these instructions carefully before beginning the installation. Failure to understand and follow installation instructions may result in injury to installer and/or end user and may void the warranty.  
If you have any questions call Sunrise Medical Technical support at 800.333.4000

**Parts included:**

Bottom Clamp  
Top Clamp  
Armtrough

4x Armtrough mount Screws  
1x Center Clamp screw  
2x Clamp screw

**Tools needed for installation:**

1x 4mm Hex Key  
1x 3mm Hex Key (supplied)


**1. Install the Armtrough Mounting Clamp (Fixed or Rotational)**

- a. Attach either the fixed top clamp (B), or the rotational hardware top clamp (C) onto your Armtrough (A) with the 4 smaller screws.

**2. Mount the Armtrough to the arm-rail.**

- a. Assemble the bottomclamp (D) and either fixed top clamp (B) or Rotational top clamp assembly (C) to the arm-rail by using the 3 clamp screws provided.

**NOTE:** . Make sure to tighten the rotational adjustment handle (E) at the preferred position before use.

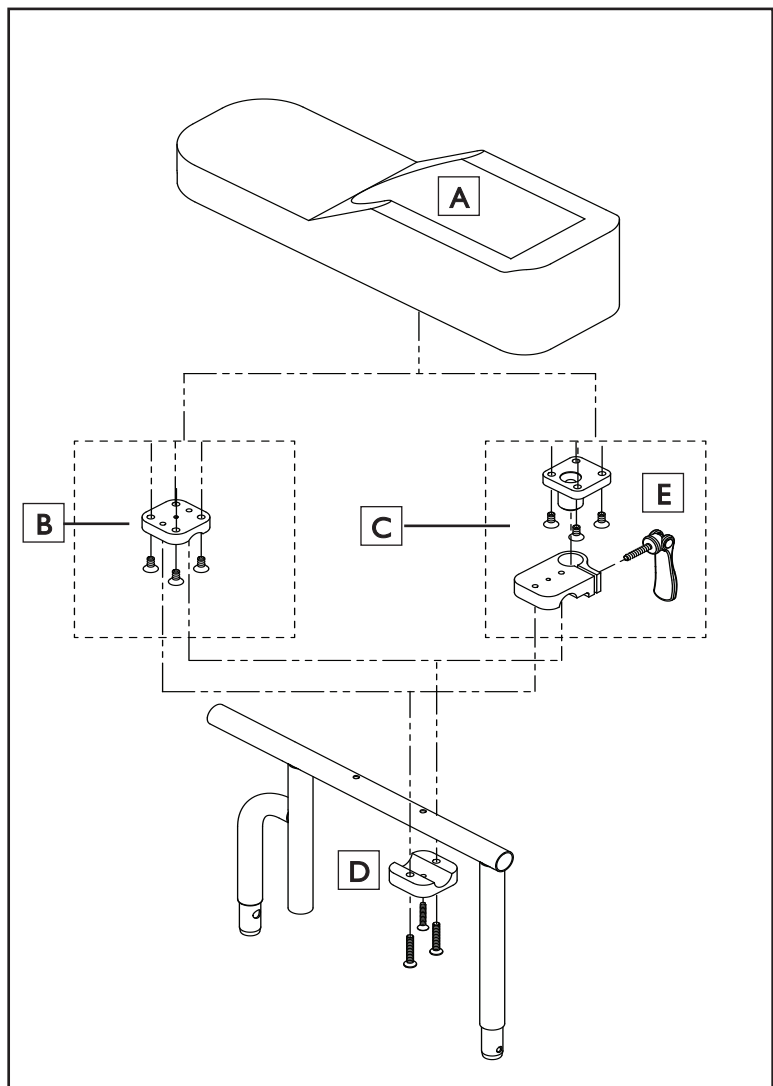

Sunrise Medical • 2842 Business Park Ave. • Fresno, CA 93727 • USA  
In Canada (800) 263-3390



Customer Service: 800.333.4000  
[www.SunriseMedical.com](http://www.SunriseMedical.com)

©2014 Sunrise Medical (US) LLC  
06.14 129294 Rev.A