



POWER RECLINE CANTILEVER ARMREST TROUGH ARMREST WIDTH ADJUSTMENT



Please read these instructions before beginning installation. Failure to understand and follow installation instructions may result in injury to installer and/or end user and may void the warranty on your Quickie wheelchair. If you have any questions call Sunrise Medical Technical support at 800.333.4000

Parts included:

1. F- Trough armrest assembly, right and/or left
2. C- 1/4-20 x 7/16" button head screws
3. D- Track, 12" for med. armrest, 14" for large
4. A- M6 x 10 socket head screws and washers

Required tools:

1. 3/16" hex key
2. 5mm hex key

A. REMOVE ARMREST 1


1. Remove hardware

- a. Remove screws and washers (A) from the bottom of cantilevered armrest (B). Remove screws (C) from track (D). Set aside hardware for reinstallation.

B. REINSTALL ARMREST 1

1. Reposition armrest

- a. Reposition track (D) to center or outside holes in bottom of armrest (E). **Note:** Fabric covering on the bottom of the armrest will need to be slit to access screw holes. Attachment plate (E) shown outside of armrest for clarity. Reattach track (D) to new armrest position using saved screws (C).
- b. Reattach trough armrest to cantilevered armrest (B) using saved screws and washers (A). Screws (A) will fit into dovetail nuts in bottom of track (D). **Note:** Lift cantilevered armrest to check for interference. Adjust as needed.

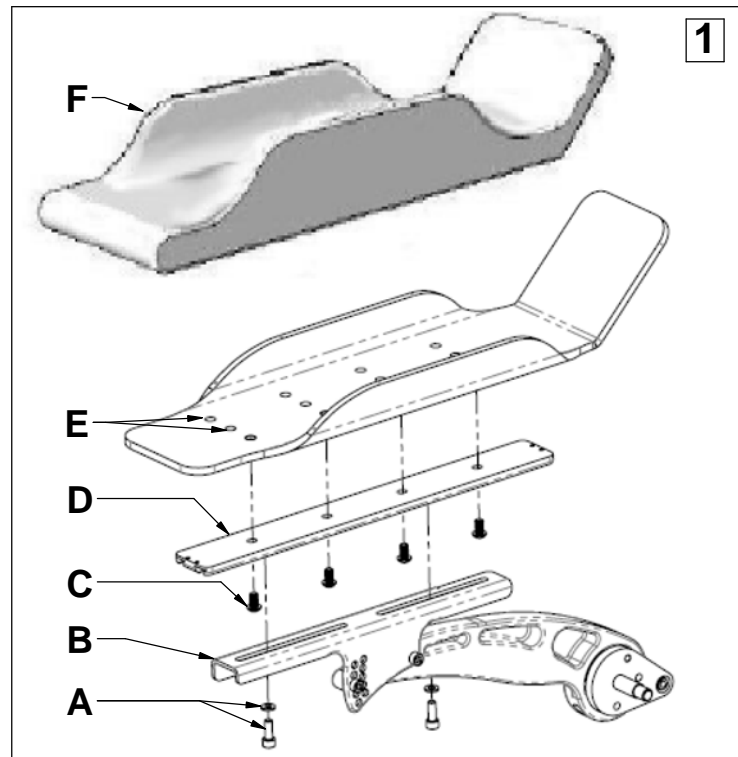

Sunrise Medical - 2842 Business Park Ave. - Fresno, CA 93727 - USA
In Canada (800) 263-3390



Customer Service: 800.333.4000
www.SunriseMedical.com

©2015 Sunrise Medical (US) LLC
11.15 MK-100243 Rev. A