
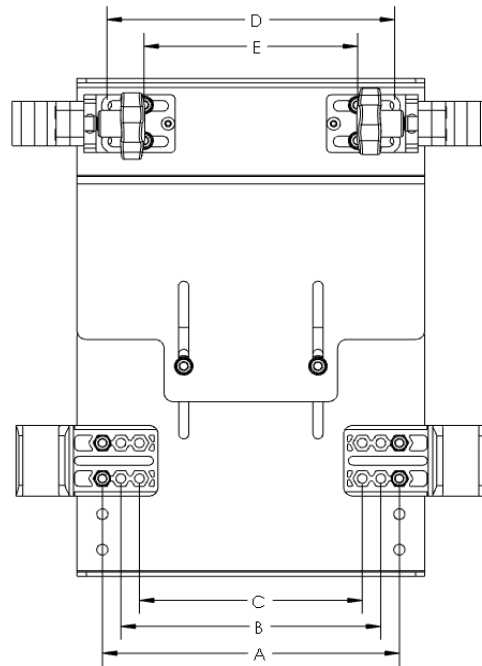


**Solid Seat for Folding Frame Only:**

Seat Pan Size	Seat Depth										L Range		
	10	11	12	13	14	15	16	17	18	19	20	Min	Max
Short	X	X	X	X								10	14
Medium					X	X						11 3/4	16 1/4
Long							X	X				13 3/4	18 1/4
X-Long									X	X	X	15 3/4	20 1/4

Colored areas indicate seat depths available for each pan size.

X = Default configurator output.

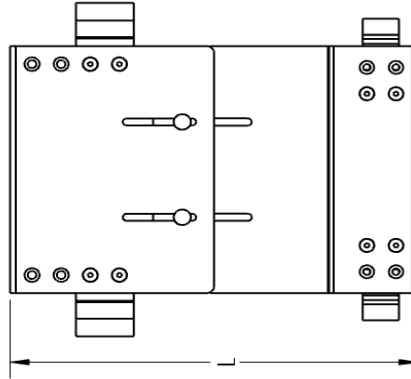

FRAME WIDTH	REAR MOUNTING	FRONT MOUNTING
10	A	E
11	B	E
12	A	E
13	B	E
14	A	E
15	B	E
16	A	E
17	B	D
18	C	D

## Standard Setup By Quickie

Definitions:

NO-GO

User Factors		Seat Frame Setup								
Seat Depth	CG Factor	Backrest Position						Hanger Receiver Position	Seat Rail Length	Seat Pan Size
		3" Fwd	2" Fwd	1" Fwd	Flush	1" Rwd	2" Rwd			
10	4.5	5	3	1	N/A	N/A	N/A	3	Sht	Sht
	5.0	6	4	2	N/A	N/A	N/A	2	Sht	Sht
	5.5	7	5	3	1	N/A	N/A	1	Sht	Sht
11	4.5	5	3	1	N/A	N/A	N/A	5	Med	Sht
	5.0	6	4	2	N/A	N/A	N/A	4	Med	Sht
	5.5	7	5	3	1	N/A	N/A	3	Sht	Sht
	6.0	8	6	4	2	N/A	N/A	2	Sht	Sht
12	6.5	9	7	5	3	1	N/A	1	Sht	Sht
	5.0	6	4	2	N/A	N/A	N/A	6	Med	Sht
	5.5	7	5	3	1	N/A	N/A	5	Med	Sht
	6.0	8	6	4	2	N/A	N/A	4	Med	Sht
12	6.5	9	7	5	3	1	N/A	3	Sht	Sht
	7.0	10	8	6	4	2	N/A	2	Sht	Sht
	5.5	7	5	3	1	N/A	N/A	7	Lng	Sht
	6.0	8	6	4	2	N/A	N/A	6	Med	Sht
13	6.5	9	7	5	3	1	N/A	5	Med	Sht
	7.0	10	8	6	4	2	N/A	4	Med	Sht
	7.5	N/A	9	7	5	3	1	3	Sht	Sht
	5.5	7	5	3	1	N/A	N/A	9	Lng	Med
14	6.0	8	6	4	2	N/A	N/A	8	Lng	Med
	6.5	9	7	5	3	1	N/A	7	Lng	Med
	7.0	10	8	6	4	2	N/A	6	Med	Med
	7.5	N/A	9	7	5	3	1	5	Med	Med
	8.0	N/A	10	8	6	4	2	4	Med	Med
15	6.5	9	7	5	3	1	N/A	9	Lng	Med
	7.0	10	8	6	4	2	N/A	8	Lng	Med
	7.5	N/A	9	7	5	3	1	7	Lng	Med
	8.0	N/A	10	8	6	4	2	6	Med	Med
15	8.5	N/A	N/A	9	7	5	3	5	Med	Med
	7.5	N/A	9	7	5	3	1	9	Lng	Lng
	8.0	N/A	10	8	6	4	2	8	Lng	Lng
	8.5	N/A	N/A	9	7	5	3	7	Lng	Lng
16	8.5	N/A	N/A	9	7	5	3	9	Lng	Lng
	9.0	N/A	N/A	10	8	6	4	6	Med	Lng
	9.5	N/A	N/A	N/A	9	7	5	7	Lng	Lng
	9.5	N/A	N/A	N/A	9	7	5	9	Lng	Xlng
17	9.5	N/A	N/A	N/A	10	8	6	8	Lng	Xlng
	10.0	N/A	N/A	N/A	10	8	6	8	Lng	Xlng

"Sht" Seat Pan Pulls Customers Seat Width and Length of 10.

"Med" Seat Pan Pulls Customers Seat Width and Length of 12.

"Lng" Seat Pan Pulls Customers Seat Width and Length of 14.

"Xlng" Seat Pan Pulls Customers Seat Width and Length of 16.

SEAT PAN SIZE:	LENGTH:
SHT	10
MED	12
LONG	14
XLNG	16

## Standard Setup by Depth

User Factors		Seat Frame Setup			Seat Pan Size
Frame Depth	CG Factor	Back Plate Position	Hanger Receiver Position	Seat Rail Length	
10	5.5	1	1	Sht	Sht
11	5.5	1	3	Sht	Sht
12	6.0	2	4	Med	Lng Frnt/Sht Rear
12	6.5	3	3	Sht	Lng Frnt/Sht Rear
13	5.5	1	7	Lng	Sht
13	6.5	3	5	Med	Sht
13	7.5	5	3	Sht	Sht
14	6.5	3	7	Lng	Med
14	7.0	4	6	Med	Med
14	8.5	7	3	Sht	Med
15	7.5	5	7	Lng	Med
15	8.5	7	5	Med	Med
15	9.5	9	3	Sht	Med
16	8.0	6	8	Lng	Lng
16	9.0	8	6	Med	Lng
17	8.5	7	9	Lng	Lng
17	10.0	10	6	Med	Lng
18	9.5	9	9	Lng	Xlng

Definitions:

GO  
NO-GO

SEAT PAN SIZE:	LENGTH:
SHT	10
MED	12
LONG	14
XLNG	16
LNG FRNT/SHRT REAR	14/10