

# GUARDIAN MICROLITE GARNET

## IN THIS SECTION


Base Frame and Shrouds .....	1.1	Handle Bar Assembly .....	1.5
Anti-Tip Wheels .....	1.2	Drive Train/Motor assembly .....	1.6
Flip Down Armrests .....	1.2	Seat and Seat frame .....	1.7
Battery and Battery Charger .....	1.3	Upper Controller .....	1.8
Front Wheel Assembly .....	1.4	Lower Controller Box .....	1.9
Rear Wheels .....	1.4	Basket .....	1.9



GARN0X02

SUPPLEMENT  
2/23/04

# BASE FRAME AND SHROUDS

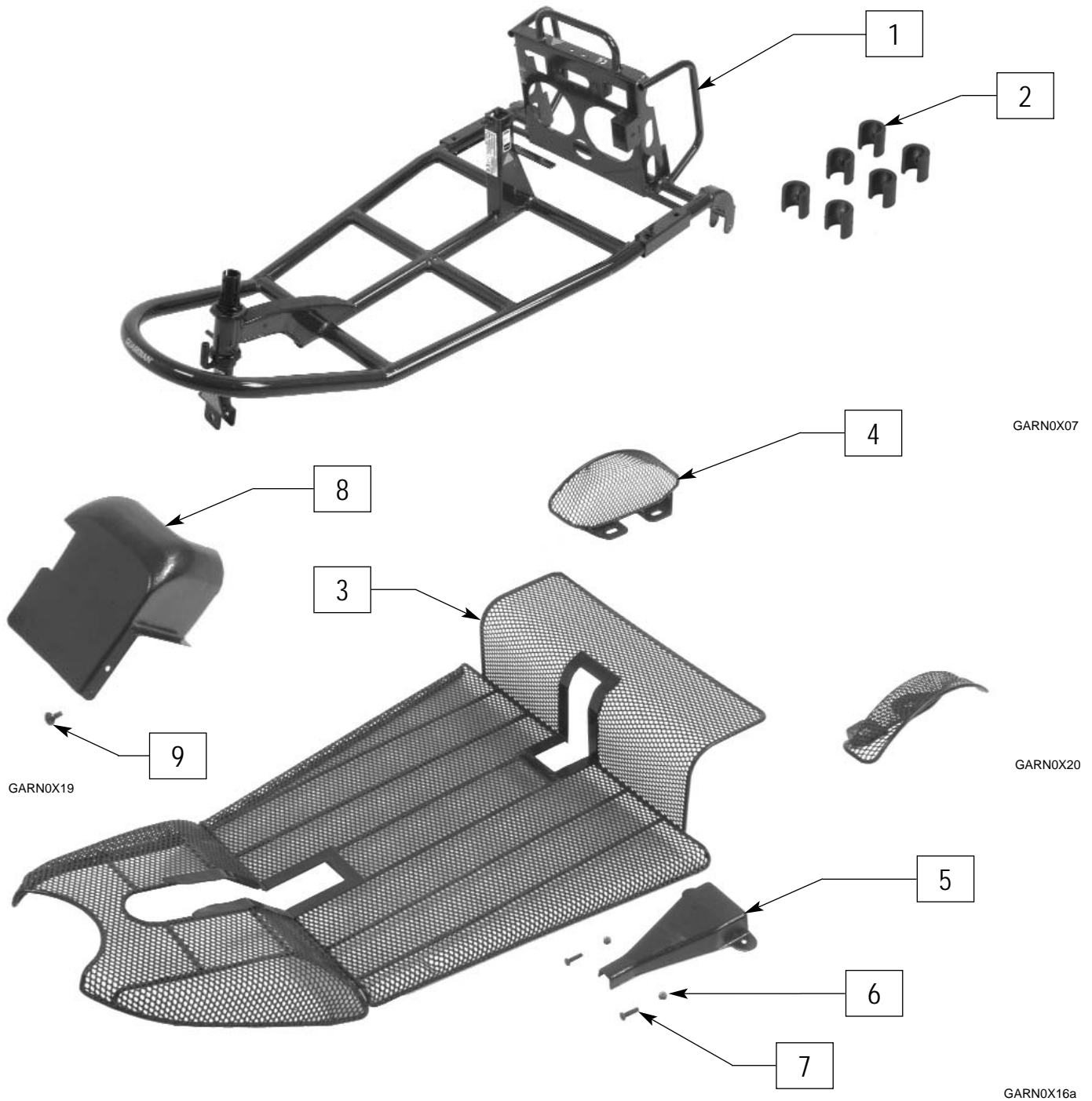

ITEM	NUMBER	DESCRIPTION	QTY
1	547004	BASE FRAME W/STEER SHAFT	
2	560372	BUMPER SET (6PC)	

ITEM	NUMBER	DESCRIPTION	QTY
1-3	662735	BASE FRAME KIT GARNET	
3-9	883094	SHROUD KIT GARNET	

## ANTI-TIP WHEELS



RUBY0X21

ITEM	NUMBER	DESCRIPTION	QTY
1-4	880987	ANTI-TIP WHEEL ASSEMBLY (PR)	



## FLIP DOWN ARMRESTS



GARN0X05


ITEM	NUMBER	DESCRIPTION	QTY
1,2	888902	ARM SET WITH KNOBS GARNET	1
2	349049	KNOB M8 x1.25 x18	2

# BATTERY AND BATTERY CHARGER

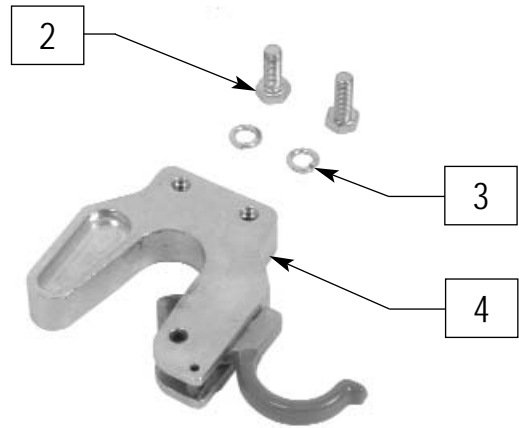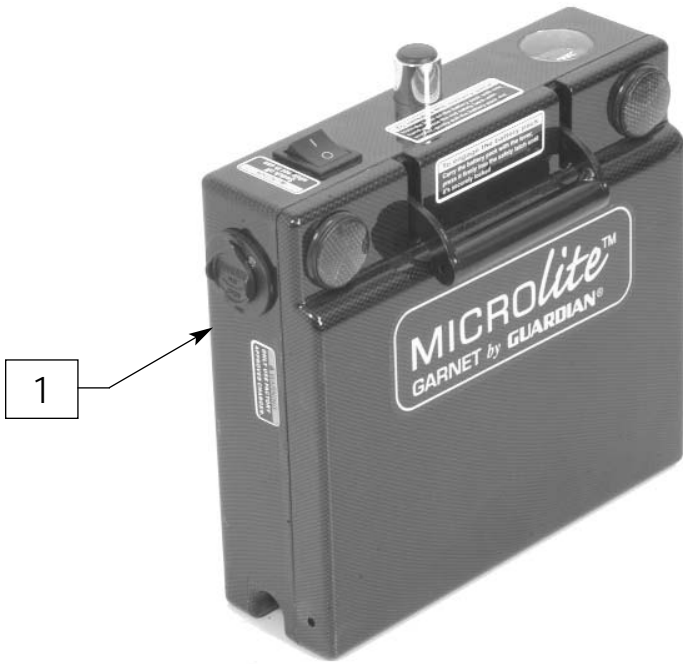


GARN0X04

GARN0X03

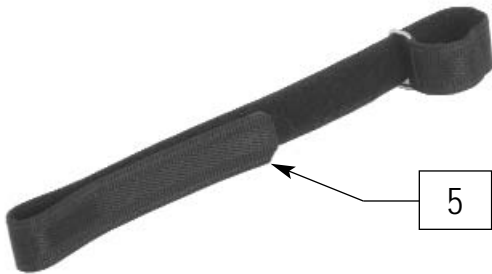

GARN0X17

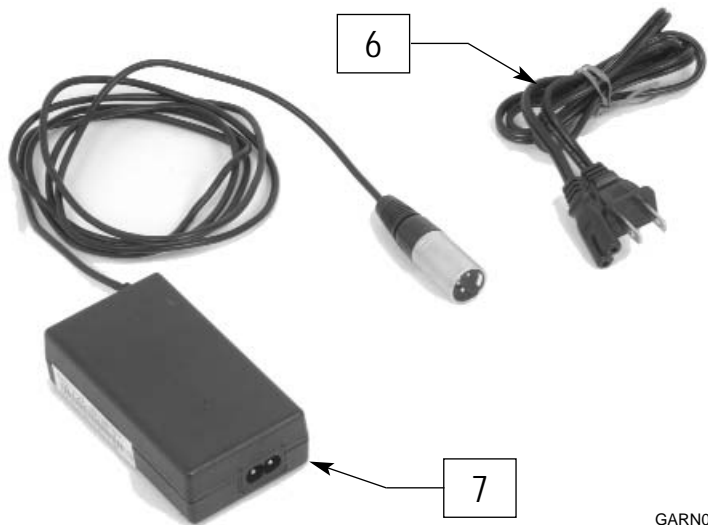


GARN0X18

ITEM	NUMBER	DESCRIPTION	QTY
1	136516	BATTERY SCOOTER MICROLITE 10A	1
2-4	888912	BRACKET KIT GARNET	2

ITEM	NUMBER	DESCRIPTION	QTY
5	901014	BATTERY HOLDER STRAP	1
6,7	888901	OFF BOARD CHARGER GARNET	1

# FRONT WHEEL ASSEMBLY

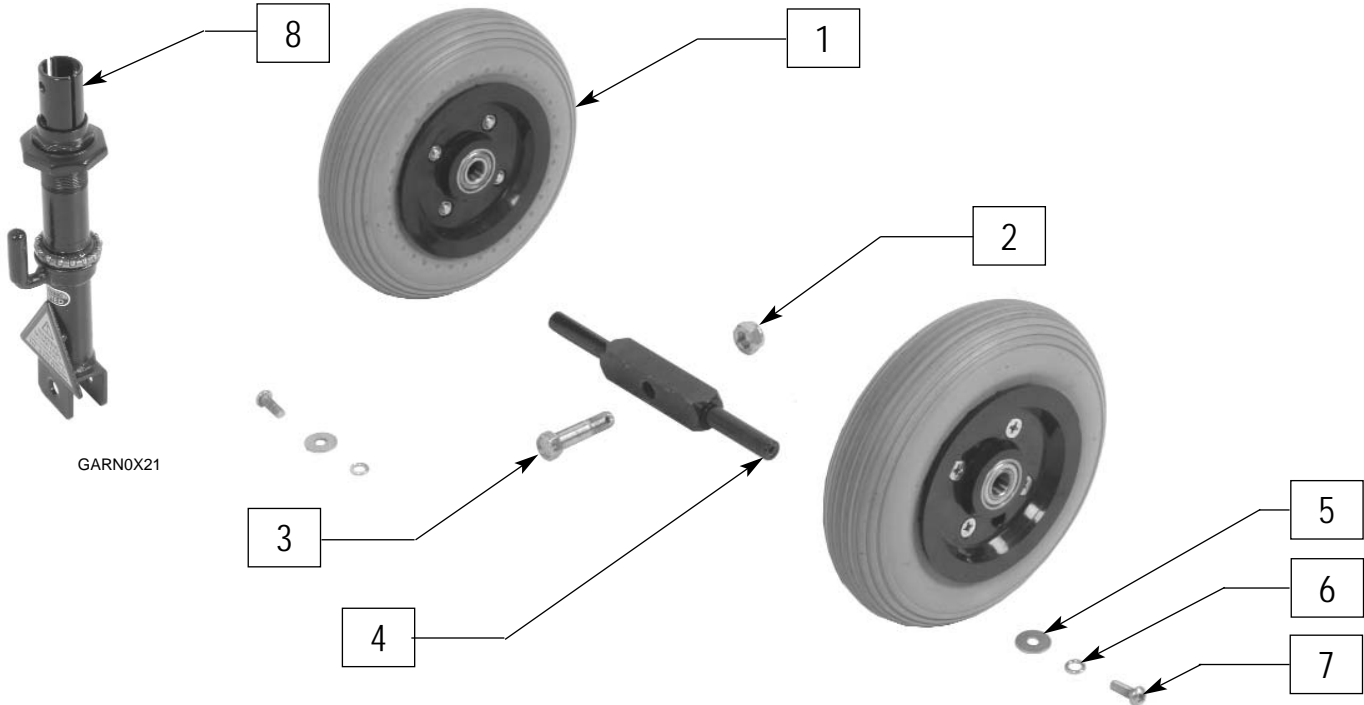


GARN0X21

GARN0X13

ITEM	NUMBER	DESCRIPTION	QTY
1	888909	FRONT WHEEL ASSY GARNET	1
1-7	888908	FRONT WHEEL KIT GARNET	

ITEM	NUMBER	DESCRIPTION	QTY
2-4,8	888916	STEERING SHAFT ASSY GARNET	


# REAR WHEELS

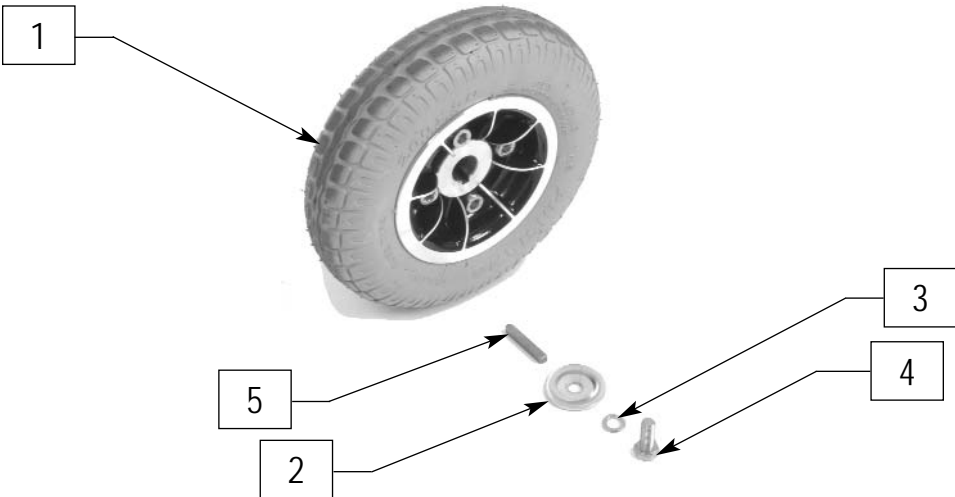

RUBY0X05

ITEM	NUMBER	DESCRIPTION	QTY
1-4	888907	REAR WHEEL KIT GARNET	


# HANDLE BAR ASSEMBLY

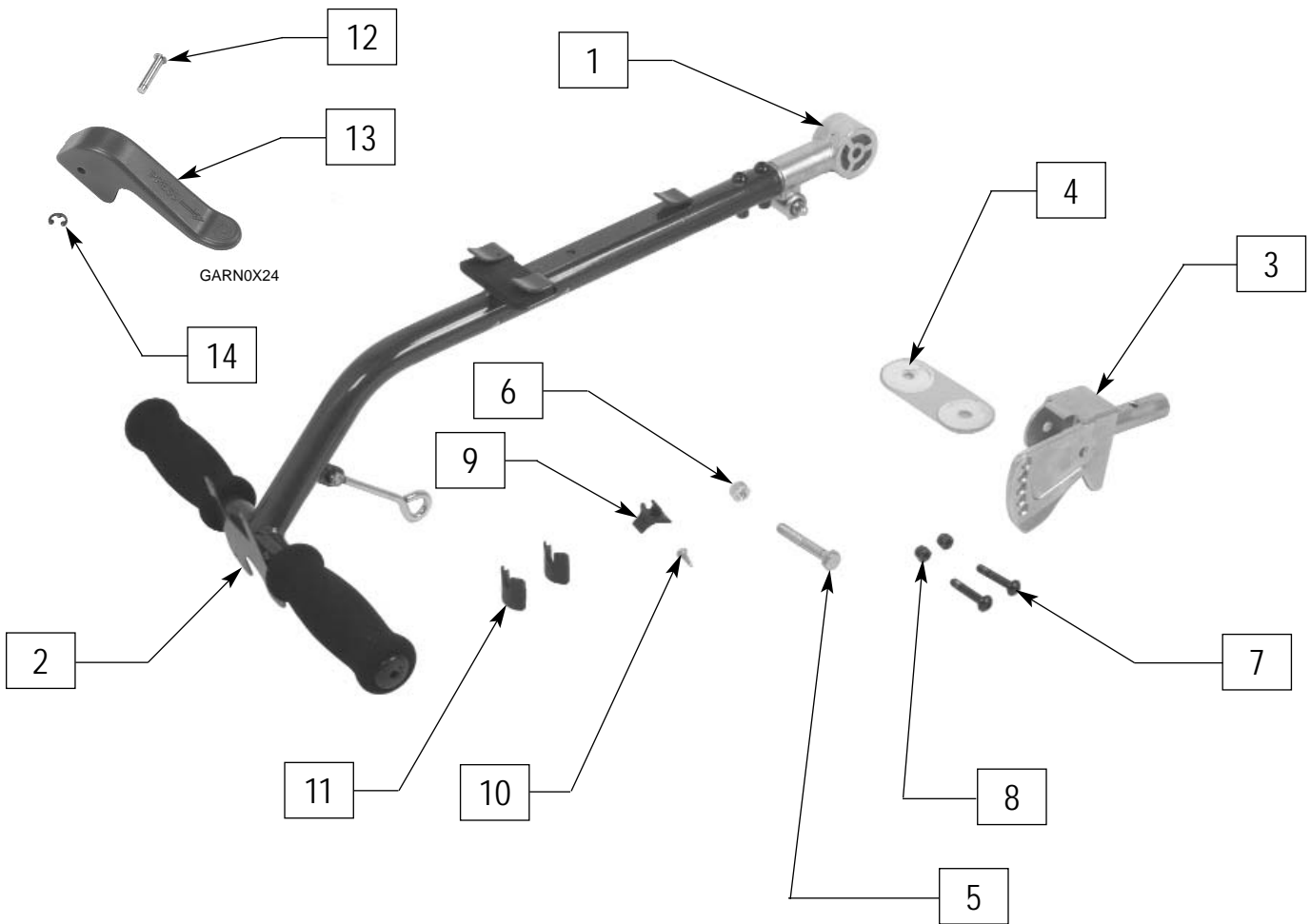

GARN0X15

ITEM	NUMBER	DESCRIPTION	QTY
1-14	888913	HANDLE/CONTROL KIT GARNET	
12-14	886985	HNDLE REL LEVER ASSM GARNET	


# DRIVE TRAIN/MOTOR ASSEMBLY

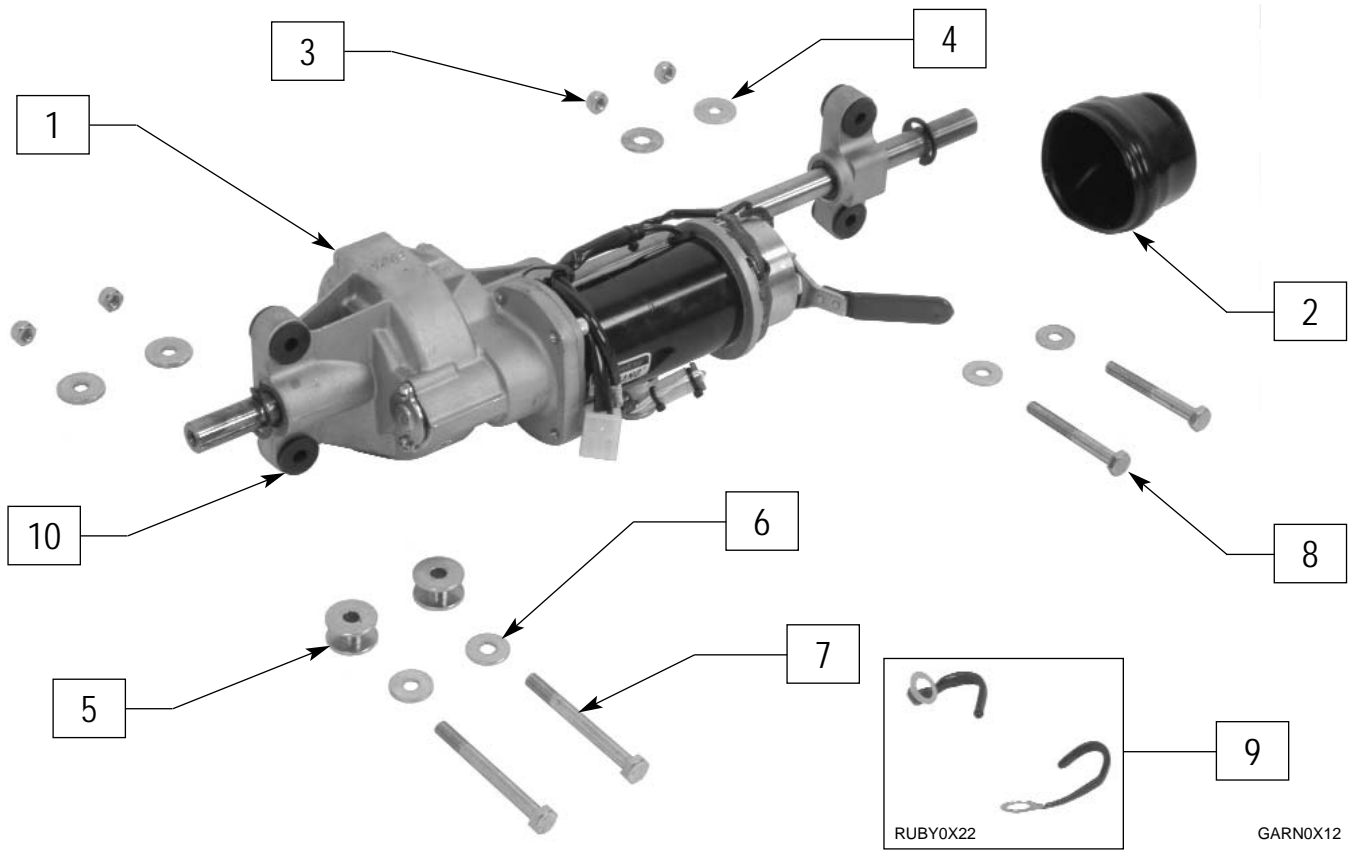

ITEM	NUMBER	DESCRIPTION	QTY
2	490016	MOTOR COVER GARNET	1
10	349602	RUBBER WASHER	8

ITEM	NUMBER	DESCRIPTION	QTY
1-10	888910	DRIVE TRAIN ASSY GARNET	

# SEAT AND SEAT FRAME

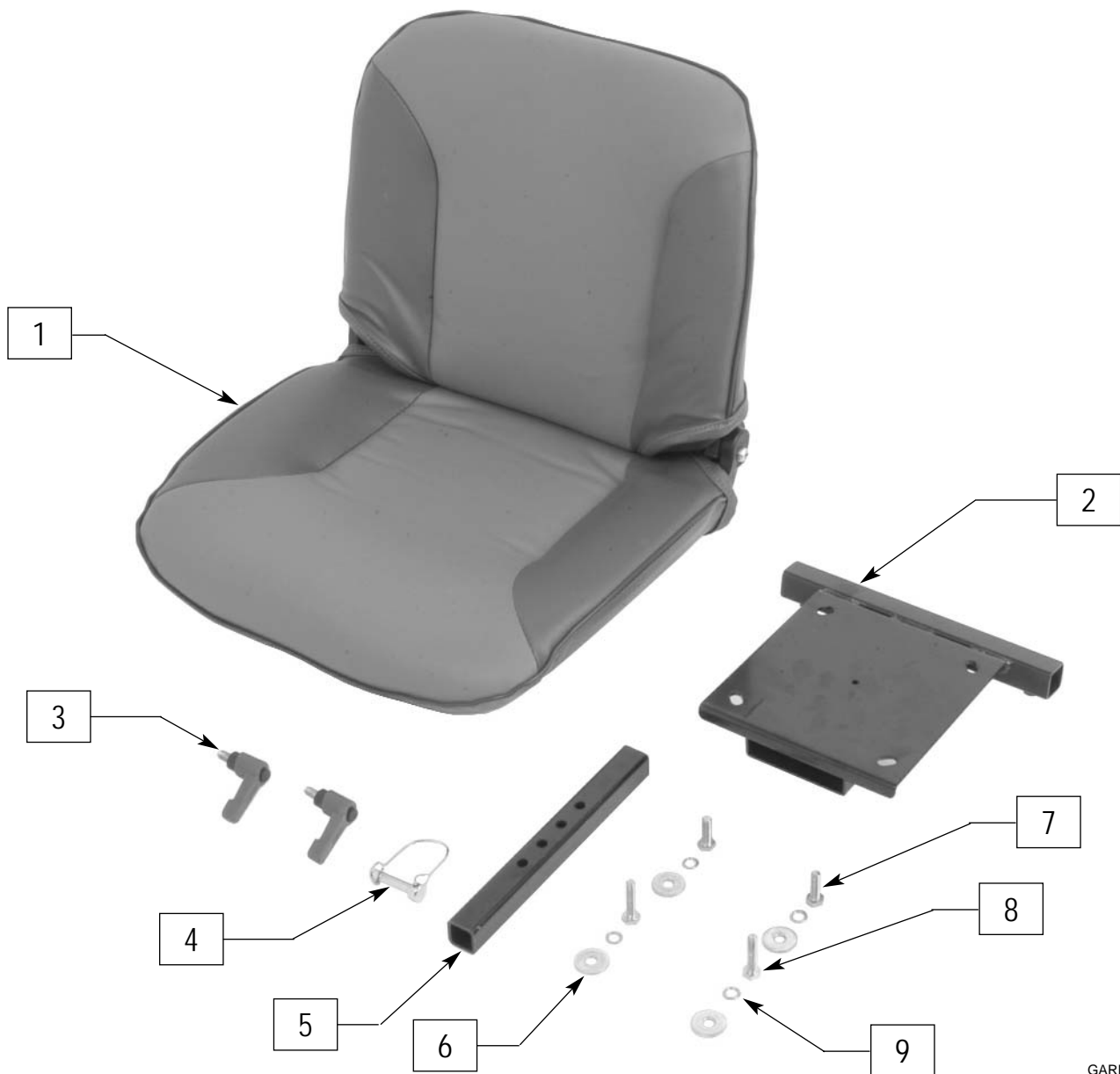


GARN0X14

ITEM	NUMBER	DESCRIPTION	QTY
1	702679	FOLD DOWN SEAT	1
3	888904	ADJUSTMENT LEVER GARNET	
4	888906	CONNECTING PIN GARNET	

ITEM	NUMBER	DESCRIPTION	QTY
4,5	888905	SEAT POST KIT GARNET	
1-9	888903	SEAT KIT GARNET	




# UPPER CONTROLLER

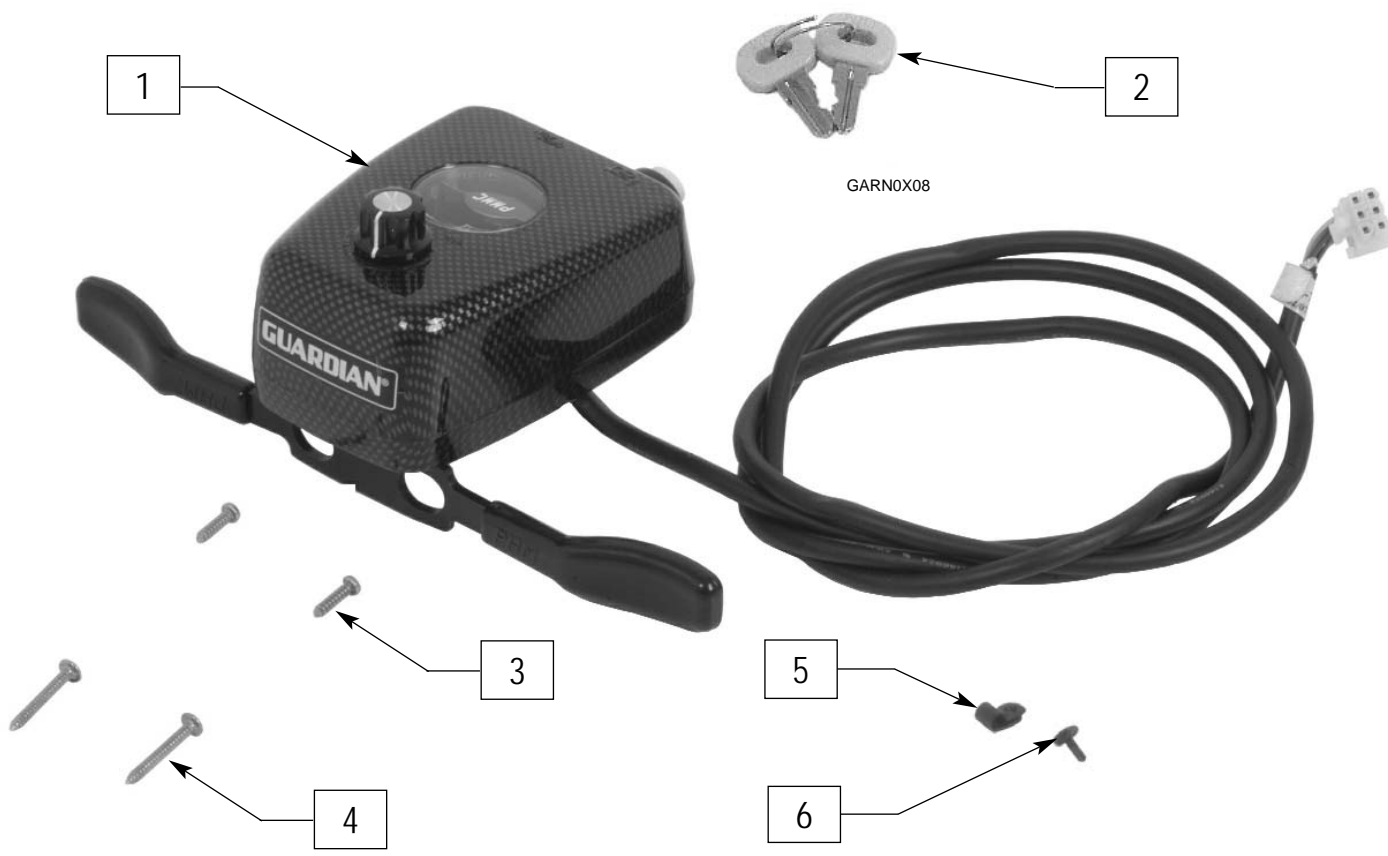

ITEM	NUMBER	DESCRIPTION	QTY
1-7	910133	UPPER CONTROLLER ASSM	

GARN0X06


## LOWER CONTROLLER BOX

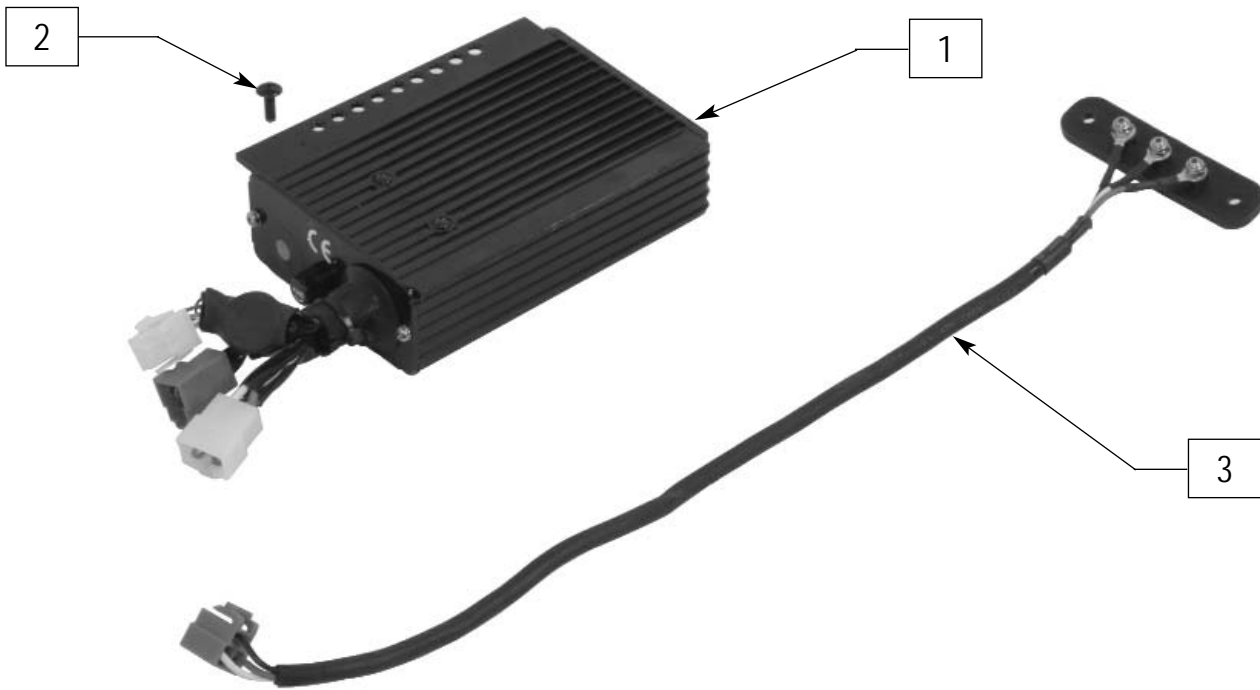

GARN0X11

ITEM	NUMBER	DESCRIPTION	QTY
1-3	888911	MAIN CONTROL BOX ASSM	

## BASKET

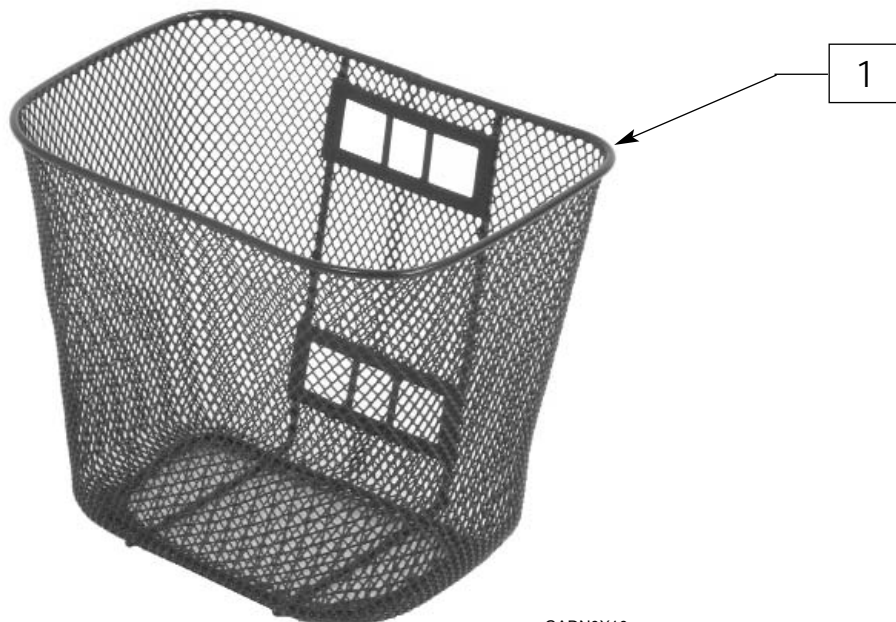

GARN0X10

ITEM	NUMBER	DESCRIPTION	QTY
1	888900	BASKET	1