

Whitmyer Biomechanix

Replacement Parts Catalog

August 2011

HEAD SUPPORT solutions



www.sunrisemedical.com




**Whitmyer
Biomechanix**


866.725.SEAT

Product	1
WHITMYER.....	3
REPLACEMENT PARTS	3
PLUSH PADS REPLACEMENT FOAM AND COVERS.....	3
SPECIALTY PLUSH AND CONTOURED CRADLE REPLACEMENT FOAM AND COVERS	4
SINGLE OCCIPITAL REPLACEMENT FOAM AND COVERS	5
S.O.F.T ADULT PADS	6
S.O.F.T PEDIATRIC PADS	7
FACIAL SUPPORT PADS.....	8
GEL INSERTS.....	10
HEAD SUPPORT SYSTEMS	11
HEAD SUPPORT MOUNTING BRACKETS	12
LATERAL FACIAL SUPPORT REPLACEMENT PARTS.....	13
ANTERIOR SUPPORTS	14
S.O.F.T BASE PLATE ASSEMBLIES	15
SUPPORT MOUNT OPTIONS.....	16
OPTIONS AND ACCESSORIES.....	17


PLUSH

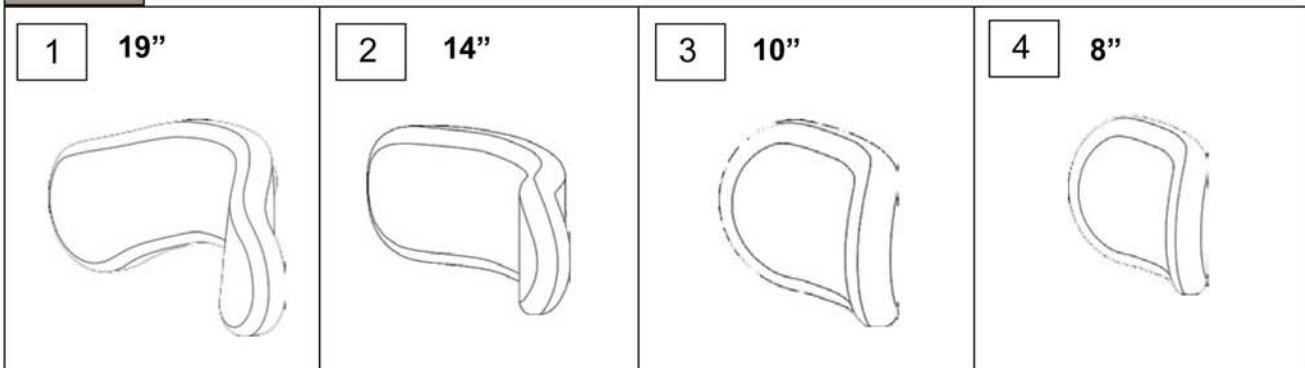


PLUSH

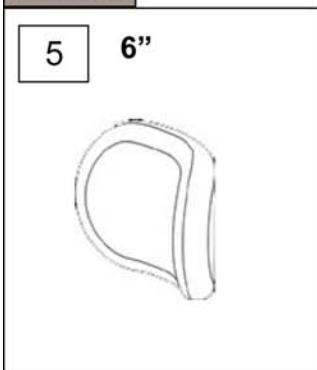

ADJUST-A-PLUSH




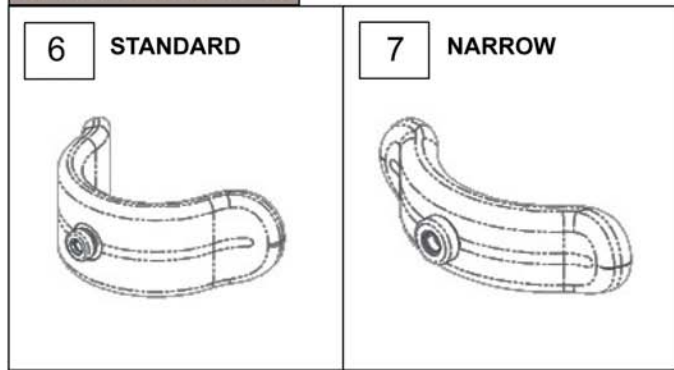


Pos.	Item Number	Description	Pos.	Item Number	Description
1	PF1900	REFOAM KIT 19"PLUSH	4	PC800V	COVER 8" PLUSH/R'VSD DARTEXX
1	PC1900L	COVER 19"PLUSH/LYCRA	4	PC800F	COVER 8"PLUSH/FLEECE
1	PC1900V	COVER 19" PLUSH/R'VSD DARTEXX	5	PF605	REFOAM KIT 6"PLUSH
1	PC1900F	COVER 19"PLUSH/FLEECE	5	PC605L	COVER 6"PLUSH/LYCRA
2	PF1400	REFOAM KIT 14"PLUSH	5	PC605V	COVER 6" PLUSH/R'VSD DARTEXX
2	PC1400L	COVER 14"PLUSH/LYCRA	5	PC605F	COVER 6"PLUSH/FLEECE
2	PC1400V	COVER 14" PLUSH/R'VSD DARTEXX	6	PF8088	FOAM STD AJUSTAPLUSH
2	PC1400F	COVER 14"PLUSH/FLEECE	6	PC8088L	COVER, STD ADJUST-A-PL, LY
3	PF1000	REFOAM KIT 10"PLUSH	6	PC8088V	COVER STD ADJ-PLUSH/RD
3	PC1000L	COVER 10"PLUSH/LYCRA	6	PC8088F	COVER STD AJUSTAPLUSH/FLEECE
3	PC1000V	COVER 10" PLUSH/R'VSD DARTEXX	7	PF8044	FOAM NARROW AJUSTAPLUSH
3	PC1000F	COVER 10"PLUSH/FLEECE	7	PC8044L	COVER, NAR ADJUST-A-PL, LY
4	PF800	REFOAM KIT 8"PLUSH	7	PC8044V	COVER, NAR ADJUST-A-PL, DR
4	PC800L	COVER 8"PLUSH/LYCRA	7	PC8044F	COVER NARROW ADJ-PLUSH/FLEECE

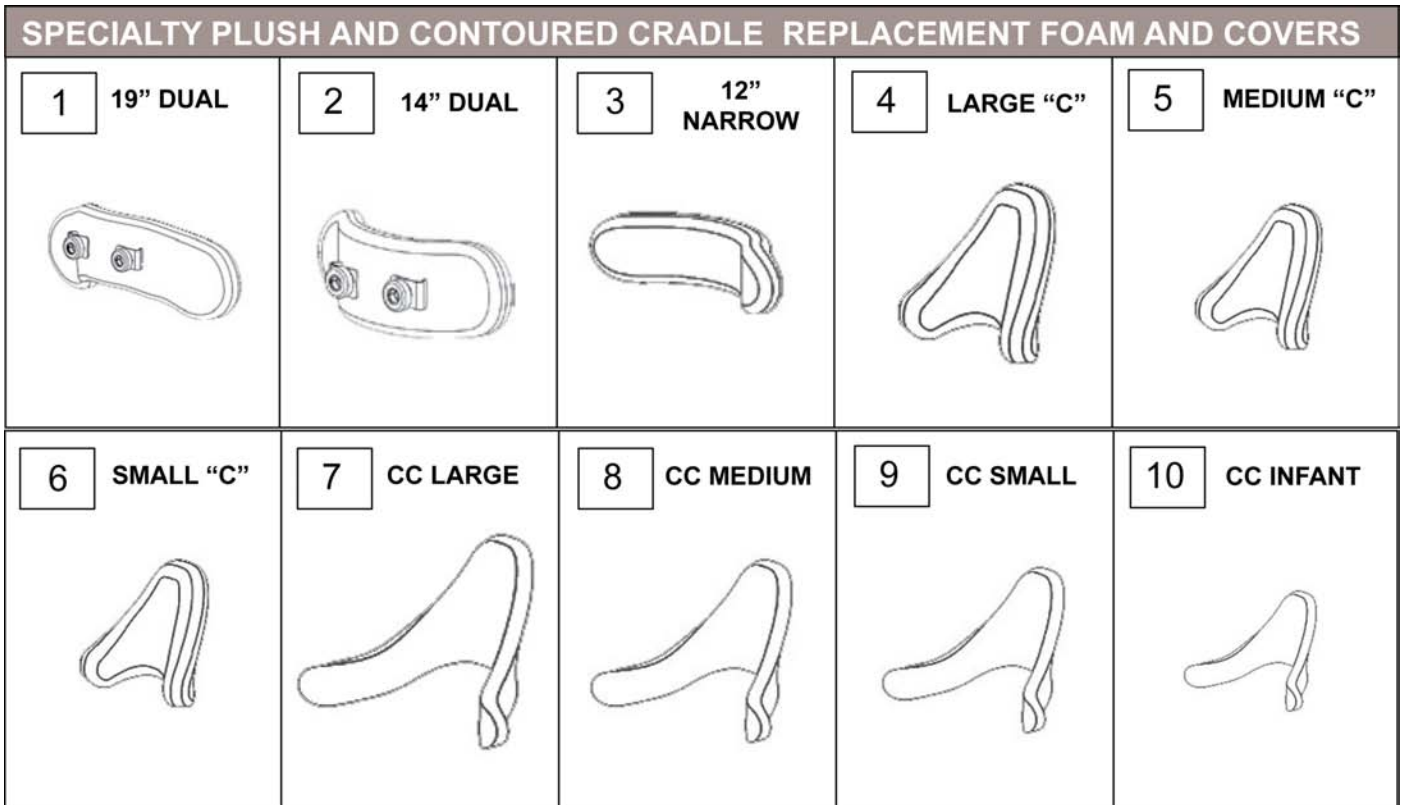


Pos.	Item Number	Description	Pos.	Item Number	Description
1	PF1900	REFOAM KIT 19"PLUSH	6	PC2010L	COVER SMALL CCPLUSH/LYCRA
1	PC1900L	COVER 19"PLUSH/LYCRA	6	PC2010V	COVER SMALL CCPLUSH/RD
1	PC1900V	COVER 19" PLUSH/R'VSD DARTEXX	6	PC2010F	COVER SMALL CCPLUSH/FLEECE
1	PC1900F	COVER 19"PLUSH/FLEECE			Contoured Cradle Pads
2	PF1400	REFOAM KIT 14"PLUSH	7	PF400	FOAM LG CONTOURED CRADLE
2	PC1400L	COVER 14"PLUSH/LYCRA	7	PC400L	COVER LG CONTED CRADLE/LYCRA
2	PC1400V	COVER 14" PLUSH/R'VSD DARTEXX	7	PC400V	COVER LG CONTRD CRADLE/RD
2	PC1400F	COVER 14"PLUSH/FLEECE	7	PC400F	COVER LG CONTED CRADLE/FLEECE
3	PF1205	REFOAM KIT 12" NARROW PLUSH	8	PF300	FOAM MED CONTOURED CRADLE
3	PC1205L	COVER NARROW PLUSH/LYCRA	8	PC300L	COVER MED CONTED CRADLE/LYCRA
3	PC1205V	CVR NARROW PLUSH/R'VSD DARTEXX	8	PC300V	COVER MED CONTRD CRADLE/RD
3	PC1205F	COVER NARROW PLUSH/FLEECE	8	PC300F	COVER MED CONTED CRADLE/FLEECE
4	PF4010	REFOAM KIT, LARGE C-PLUSH	9	PF200	FOAM SMALL CONTOURED CRADLE
4	PC4010L	COVER LRG CCPLUSH/LYCRA	9	PC200L	COVER SML CONTED CRADLE/LYCRA
4	PC4010V	COVER LRG CCPLUSH/RD	9	PC200V	COVER SML CONTRD CRADLE/RD
4	PC4010F	COVER LRG CCPLUSH/FLEECE	9	PC200F	COVER SML CONTED CRADLE/FLEECE
5	PF3010	REFOAM KIT, MED C-PLUSH	10	INFTF200	FOAM INFANT CONTOURED CRADLE
5	PC3010L	COVER MED CCPLUSH/LYCRA	10	INFTC200-L	COVER INFANT CONTED CRADLE/LYCRA
5	PC3010V	COVER MED CCPLUSH/RD	10	INFTC200-V	COVER INFANT CONTRD CRADLE/RD
5	PC3010F	COVER MED CCPLUSH/FLEECE	10	INFTC200F	COVER INFANT CONTED CRADLE/FLEECE
6	PF2010	REFOAM KIT, SMALL C-PLUSH			

SINGLE OCCIPITAL REPLACEMENT FOAM AND COVERS

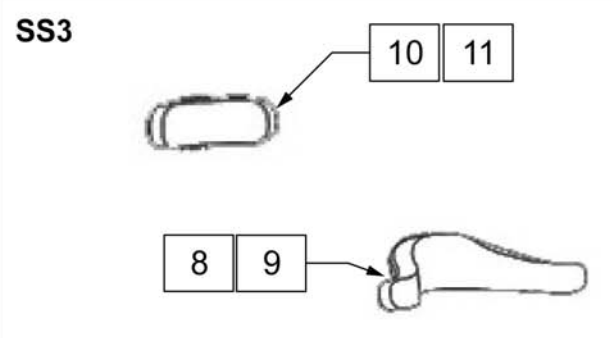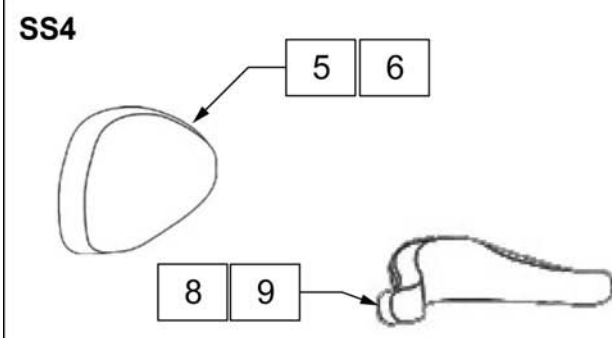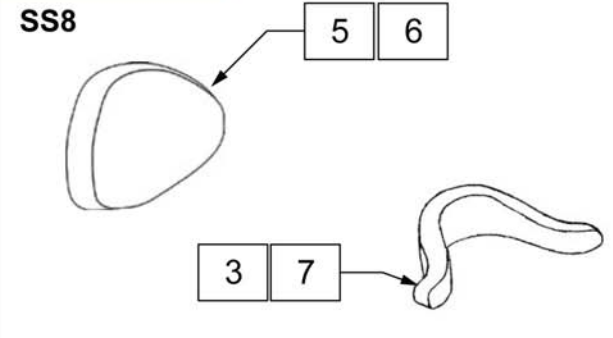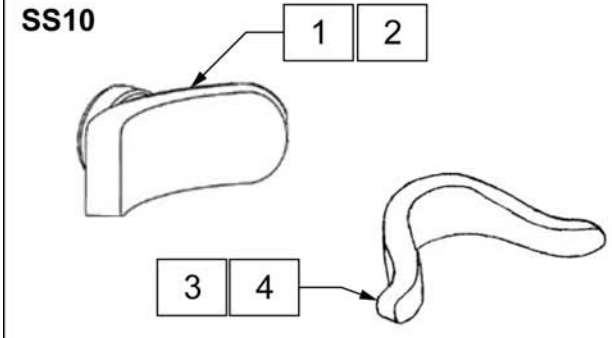


Pos.	Item Number	Description	Pos.	Item Number	Description
1	PF1000	REFOAM KIT 10"PLUSH	1,3	PFSS10	COVER OCC PED/2 1/4X5 LAT/RD
2	PC1000L	COVER, 10" PLUSH, LYCRA	2,4	PCSS10L	REFOAM KIT SOFT-SS10 CONFIG
2	PC1000V	COVER, 10" PLUSH, DR	2,4	PCSS10V	COVER SET SOFT-SS10/LYCRA
3	PFSS01	COVER LRG SINGLE SUB-OCC/LYCRA	2,4	PCSS10F	COVER SET SOFT-SS10/DR
4	PCSSO1L	REFOAM KIT LRG SINGLE SUB-OCC	3,5	PFSS8	COVER SET SOFT-SS10/FLEECE
4	PCSSO1V	COVER LRG SINGLE SUB-OCC/RD	5,8	PFSS4	REFOAM KIT SOFT SS8
5	PF600	REFOAM KIT ADULT OCC/4"X 6"LAT	6,7	PCSS8L	COVER SET SOFT-SS8/LYCRA
6	PC600L	COVER, ADULT OCCIPITAL/LYCRA	6,7	PCSS8V	COVER SET SOFT-SS8/DR
6	PC600V	COVER ADULT OCC/R'VSD DARTEXX	6,7	PCSS8F	COVER SET SOFT-SS8/FLEECE
7	PCSSO1L	COVER LRG SINGLE SUB-OCC/LYCRA	6,9	PCSS4L	REFOAM KIT PRO-SS4
7	PCSSO1V	COVER LRG SINGLE SUB-OCC/RD	6,9	PCSS4V	COVER SET SOFT-SS4/LYCRA
8	PFSSO3	REFOAM KIT PED SINGLE SUB-OCC	6,9	PCSS4F	COVER SET SOFT-SS4/DR
9	PCSSO3L	REFOAM KIT PED SINGLE SUB-OCC	8,10	PFSS3	COVER SET SOFT-SS4/FLEECE
9	PCSSO3V	COVER SINGLE SUB-OCC/LYCRA	9,11	PCSS3L	REFOAM KIT SOFT SS3
10	PF500	COVER SML SINGLE SUB-OCC/RD	9,11	PCSS3V	COVER SET SOFT-SS3/LYCRA
11	PC500L	REFOAM KIT PED REAR/STD LAT	9,11	PCSS3F	COVER SET SOFT-SS3/DR
11	PC500V	COVER PED OCCIPITAL/LYCRA			

S.O.F.T ADULT PADS

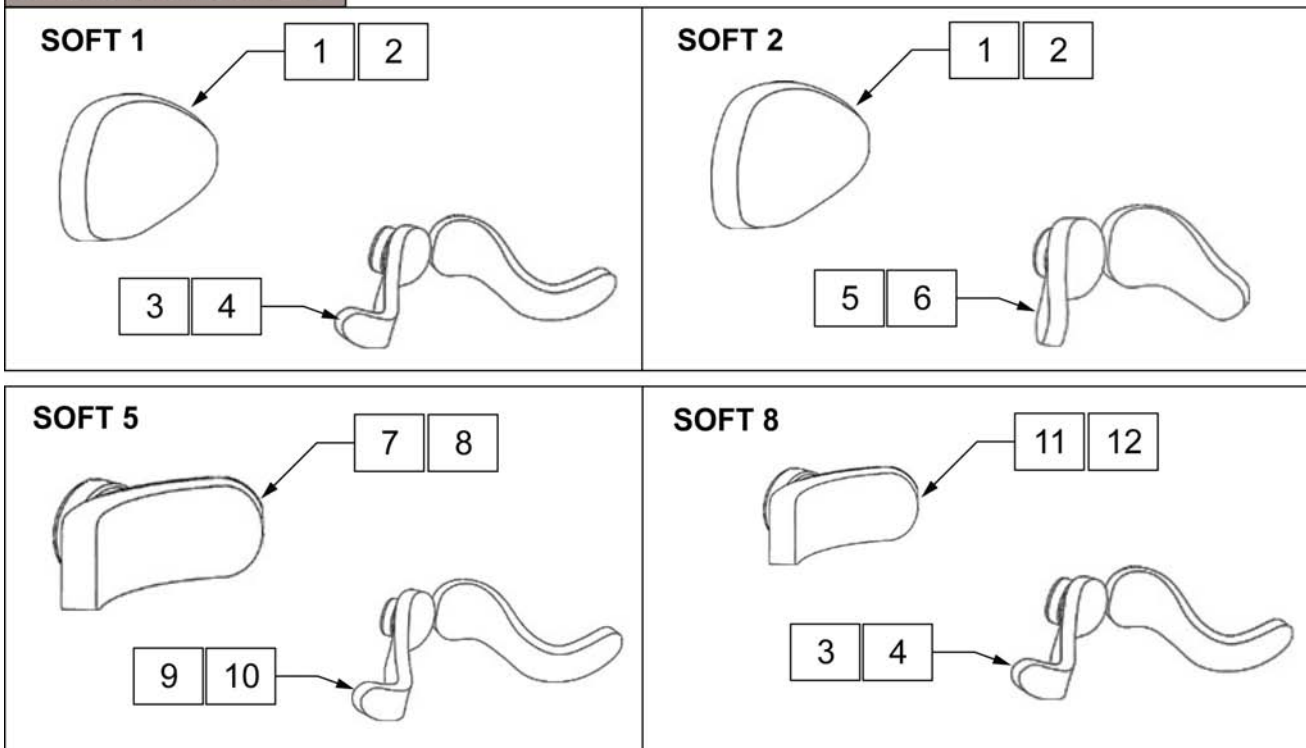


Pos.	Item Number	Description	Pos.	Item Number	Description
1	PF600	REFOAM KIT ADULT OCC/4"X 6"LAT	10	PCSO5RV	COVER RIGHT SZ5 SUB-OCC/RD
2	PC600L	COVER, ADULT OCCIPITAL/LYCRA	11	PF800	REFOAM KIT 8"PLUSH
2	PC600V	COVER ADULT OCC/R'VSD DARTEXX	12	PC800L	COVER, 8" PLUSH, LYCRA
3	PFSO1L	REFOAM KIT LFT SIZE1 SUB-OCC	12	PC800V	COVER, 8" PLUSH, DR
3	PFSO1R	REFOAM KIT LFT SIZE1 SUB-OCC	1,3	PF1	COVER LEFT SZ1 SUB-OCC/LYCRA
4	PCSO1LL	COVER LEFT SZ1 SUB-OCC/LYCRA	1,5	PF2	COVER RIGHT SZ1 SUB-OCC/LYCRA
4	PCSO1RL	COVER RIGHT SZ1 SUB-OCC/LYCRA	2,4	PC1L	COVER LEFT SZ1 SUB-OCC/RD
4	PCSO1LV	COVER LEFT SZ1 SUB-OCC/RD	2,4	PC1V	REFOAM KIT SOFT-1 PAD CONFIG
4	PCSO1RV	COVER RIGHT SZ1 SUB-OCC/RD	2,4	PC1F	COVER SET SOFT-1/LYCRA
5	PFSO2L	REFOAM KIT LFT SIZE2 SUB-OCC	2,6	PC2L	COVER SET SOFT-1 /VINYL
5	PFSO2R	REFOAM KIT RIGHT SIZE2 SUB-OCC	2,6	PC2V	COVER SET SOFT-1/FLEECE
6	PCSO2LL	COVER LEFT SZ2 SUB-OCC/LYCRA	2,6	PC2F	REFOAM KIT SOFT-2 PAD CONFIG
6	PCSO2RL	COVER RIGHT SZ2 SUB-OCC/LYCRA	7,9	PF5	COVER SET SOFT-2/LYCRA
6	PCSO2LV	COVER LEFT SZ2 SUB-OCC/RD	8,10	PC5L	COVER SET SOFT-2 /VINYL
6	PCSO2RV	COVER RIGHT SZ2 SUB-OCC/RD	8,10	PC5V	COVER SET SOFT-2/FLEECE
7	PF1000	REFOAM KIT 10"PLUSH	8,10	PC5F	PART NOT CREATED
9	PFSO5L	COVER, 10" PLUSH, LYCRA	3,11	PF8	COVER SET SOFT-5/LYCRA
9	PFSO5R	COVER, 10" PLUSH, DR	4,12	PC8L	COVER SET SOFT-5 /VINYL
8	PC1000L	REFOAM KIT LFT SIZE5 SUB-OCC	4,12	PC8V	COVER SET SOFT-5/FLEECE
8	PC1000V	REFOAM KIT RIGHT SIZE5 SUB-OCC	4,12	PC8F	REFOAM KIT SOFT-8 PAD CONFIG
10	PCSO5LL	COVER LEFT SZ5 SUB-OCC/LYCRA	4,12	PC8L	COVER SET SOFT-8/LYCRA
10	PCSO5RL	COVER RIGHT SZ5 SUB-OCC/LYCRA	4,12	PC8V	COVER SET SOFT-8 /VINYL
10	PCSO5LV	COVER LEFT SZ5 SUB-OCC/RD	4,12	PC8F	COVER SET SOFT-8/FLEECE



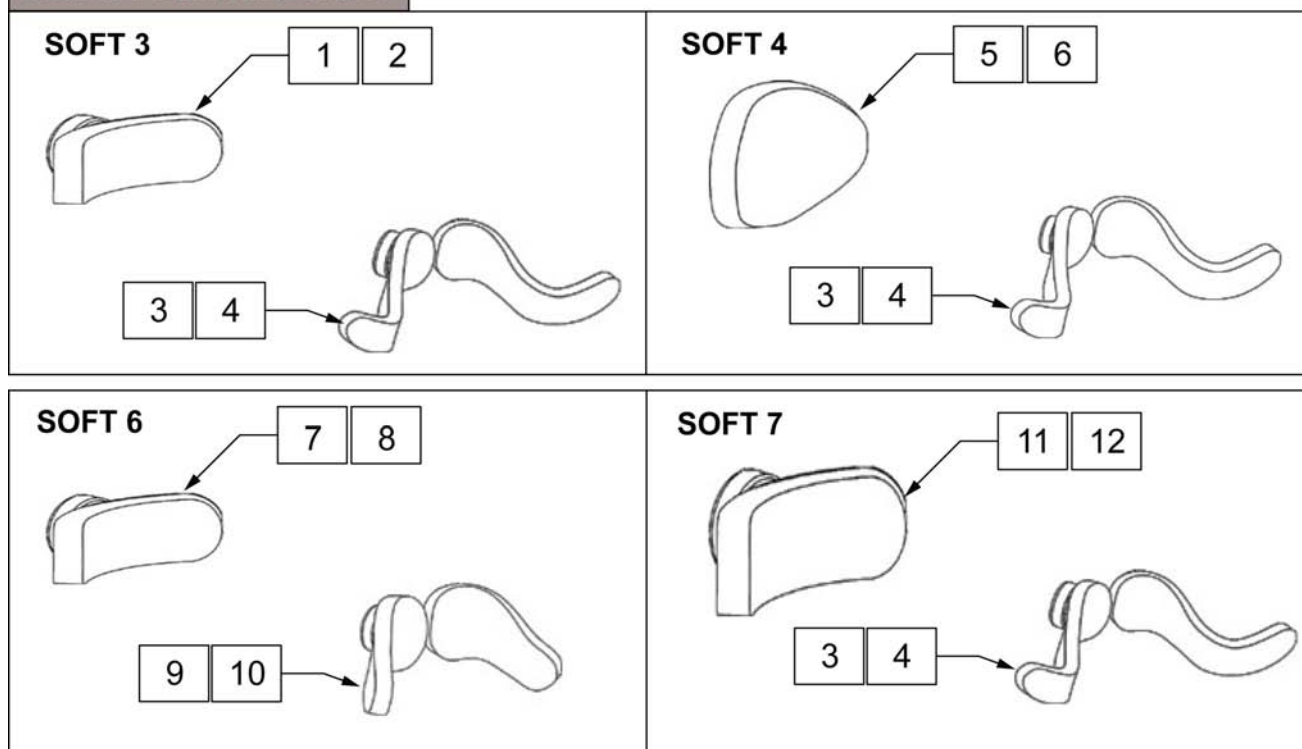

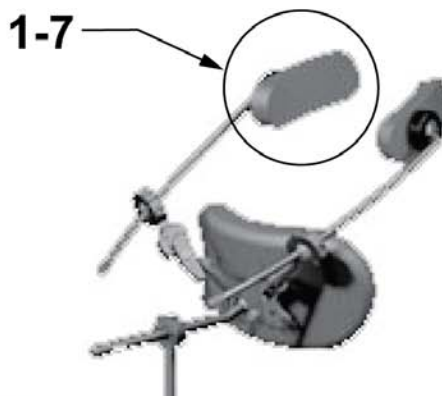


S.O.F.T PEDIATRIC PADS



Pos.	Item Number	Description	Pos.	Item Number	Description
1	PF500	REFOAM KIT SOFT-3 PAD CONFIG	11	PF800	COVER, ADULT OCCIPITAL/LYCRA
2	PC500L	COVER SET SOFT-3/LYCRA	12	PC800L	COVER ADULT OCC/R'VSD DARTEXX
2	PC500V	COVER SET SOFT-3 /VINYL	12	PC800V	COVER LEFT SZ3 SUB-OCC/LYCRA
3	PFSO3L	COVER SET SOFT-3/FLEECE	1,3	PF3	COVER RIGHT SZ3 SUB-OCC/LYCRA
3	PFSO3R	REFOAM KIT PED REAR/STD LAT	2,4	PC3L	COVER LEFT SZ3 SUB-OCC/RD
4	PCSO3LL	REFOAM KIT LFT SIZE3 SUB-OCC	2,4	PC3V	COVER RIGHT SZ3 SUB-OCC/RD
4	PCSO3RL	REFOAM KIT RIGHT SIZE3 SUB-OCC	2,4	PC3F	REFOAM KIT SOFT-6 PAD CONFIG
4	PCSO3LV	COVER PED OCCIPITAL/LYCRA	3,5	PF4	COVER SET SOFT-6/LYCRA
4	PCSO3RV	COVER OCC PED/2 1/4X5 LAT/RD	3,11	PF7	COVER SET SOFT-6 /VINYL
5	PF600	COVER LEFT SZ3 SUB-OCC/LYCRA	4,6	PC4L	COVER SET SOFT-6/FLEECE
6	PC600L	COVER RIGHT SZ3 SUB-OCC/LYCRA	4,6	PC4V	FOAM PED OCC / 2 1/4"X 5"LAT
6	PC600R	COVER LEFT SZ3 SUB-OCC/RD	4,6	PC4F	FOAM PRO LFT/RGT SZ 4 SUB-OCC
7	MF020310	COVER RIGHT SZ3 SUB-OCC/RD	4,12	PC7L	COVER PED OCCIPITAL/LYCRA
9	MF020252	REFOAM KIT SOFT-4 PAD CONFIG	4,12	PC7V	COVER OCC PED/2 1/4X5 LAT/RD
8	PC500L	COVER SET SOFT-4/LYCRA	4,12	PC7F	COVER LEFT SZ4 SUB-OCC/LYCRA
8	PC500V	COVER SET SOFT-4 /VINYL	7,9	PF6	COVER RIGHT SZ4 SUB-OCC/LYCRA
10	PCSO4LL	COVER SET SOFT-4/FLEECE	8,10	PC6L	COVER LEFT SZ4 SUB-OCC/RD
10	PCSO4RL	REFOAM KIT ADULT OCC/4"X 6"LAT	8,10	PC6V	COVER RIGHT SZ4 SUB-OCC/RD
10	PCSO4LV	REFOAM KIT LFT SIZE3 SUB-OCC	8,10	PC6F	REFOAM KIT SOFT-7 PAD CONFIG
10	PCSO4RV	REFOAM KIT RIGHT SIZE3 SUB-OCC			




LATERAL FACIAL SUPPORT PADS

1	SPOT	2	STD	3	LONG	4	OVAL
							
5	LARGE-C	6	MEDIUM-C	7	SMALL-C		
							

Pos.	Item Number	Description
1	PL10L	LF PAD, SPOT, LYCRA
1	PL10V	LF PAD, SPOT, DR
2	PL15L	LF PAD, STANDARD, LYCRA
2	PL15V	LF PAD, STANDARD, DR
3	PL20L	LF PAD, LONG, LYCRA
3	PL20V	LF PAD, LONG, DR
4	P600L	OCCIP PAD, ADULT, LYCRA
4	P600V	OCCIP PAD, ADULT, DR
5	PL30L	LF PAD, LARGE C, LYRCA
5	PL30V	LF PAD, LARGE C, DR
6	PL35L	LF PAD, MED C, LYCRA
6	PL35V	LF PAD, MED C, DR
7	PL40L	LF PAD, SM C, LYCRA
7	PL40V	LF PAD, SM C, DR


S.O.F.T GEL PAD INSERTS

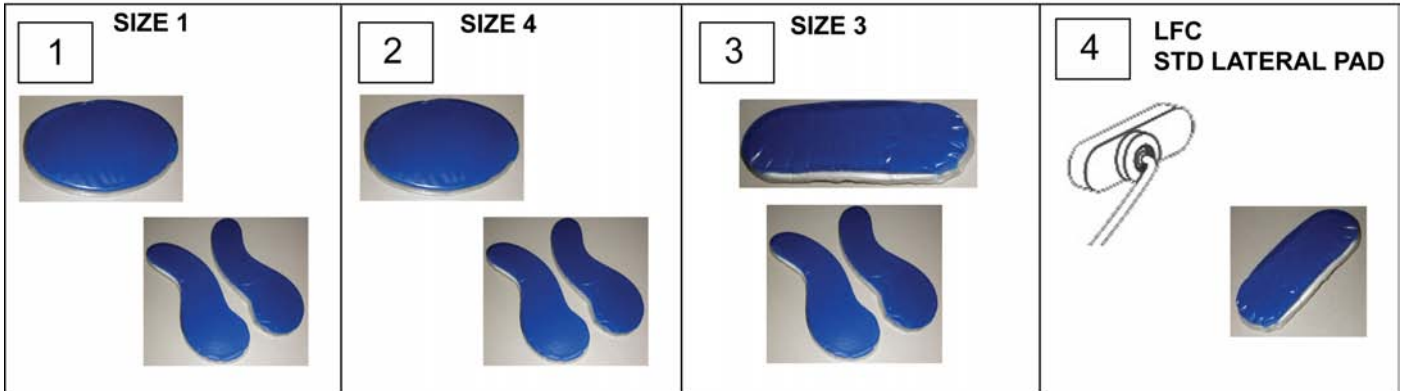

Pos.	Item Number	Description
1	PG1	GEL INSERTS SOFT-1
2	PG4	GEL INSERTS SOFT-4
3	PG3	GEL INSERTS SOFT-3
4	PG500	GEL INSERT PED OCC/STD LAT


HEAD SUPPORT SYSTEMS

HEAD SUPPORTS

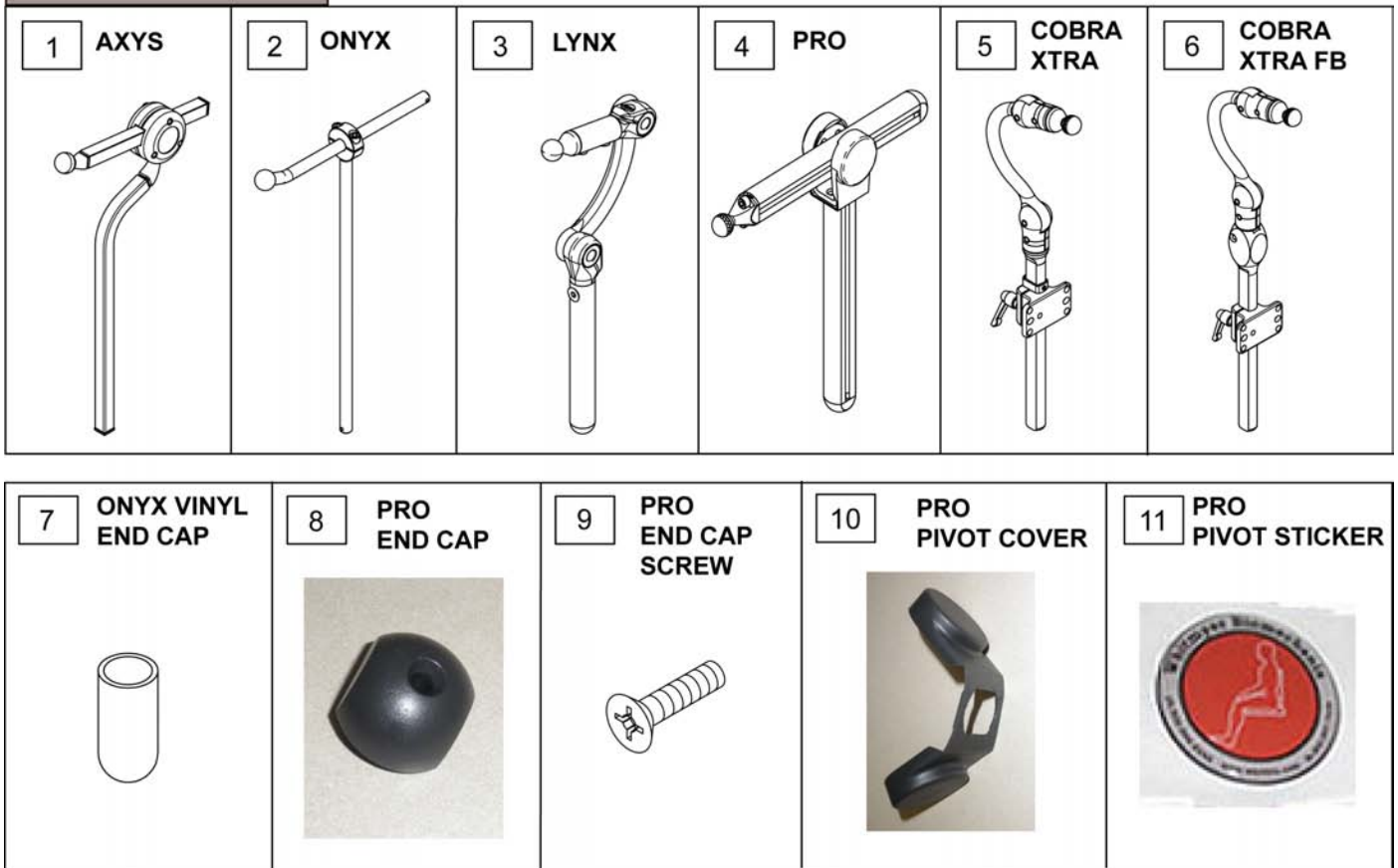


Pos.	Item Number	Description
1	AX7010	AXYS CORE MOUNT HDW
2	OX7010	ONYX CORE MOUNT HDW, ADULT
3	LX7000	LINX CORE MOUNT HDW
4	PRC7010	PRO CORE MOUNT HDW, ADULT
5	CBRA7010	COBRA XTRA MOUNT HDW, ADUL
5	CBRA7030	COBRA XTRA MOUNT HDW, PED
5	CBRA7080	COBRA XTRA MOUNT HDW, EI
6	CBRA5010	COBRA XTRA FB MOUNT HDW, ADUL
6	CBRA5030	COBRA XTRA FB MOUNT HDW, PED
6	CBRA5080	COBRA XTRA FB MOUNT HDW, EI
7	MW000800	ENDCAP 7/16ID VINYL BLK
8	PRC2110	ENDCAP, PRO ABS, BLK
9	MF014765	6-32 X 1 FH PH MS (END CAP)
10	PRC2122	PLAS HORIZONTAL PIVOT, BLK
11	MF014925	STICKER DOMED

DETACHABLE

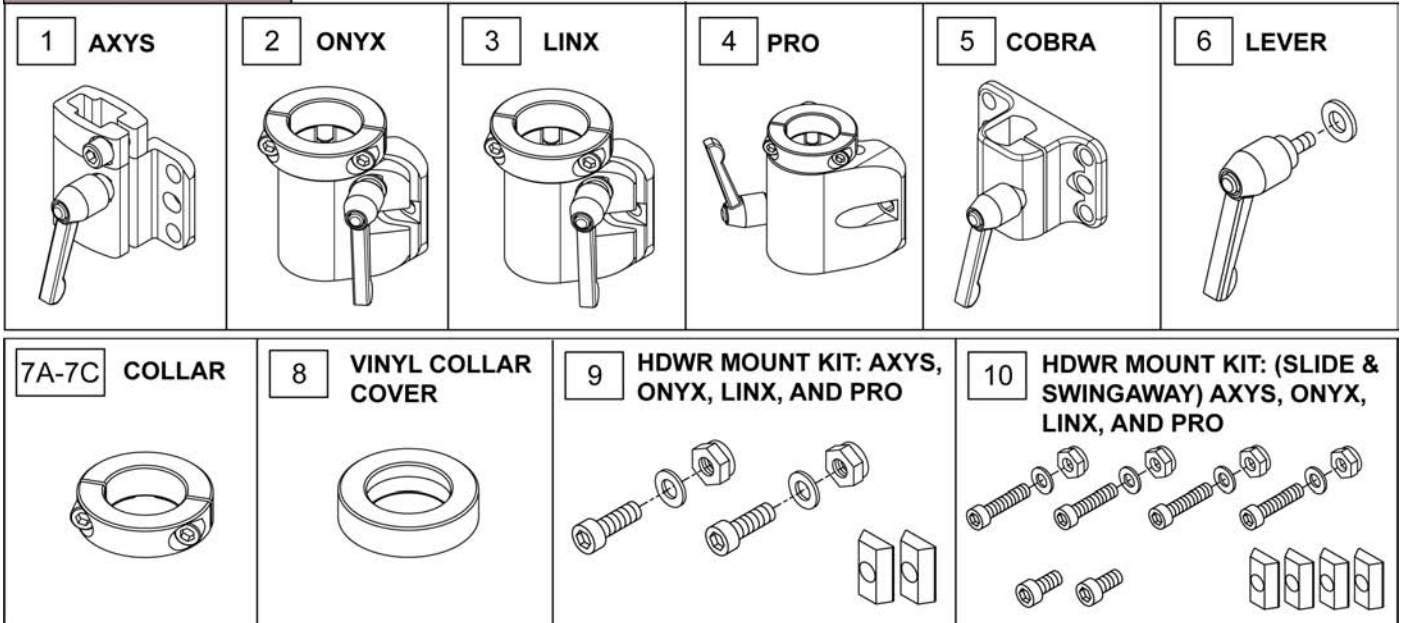


MOUNTING INTERFACE OPTIONS

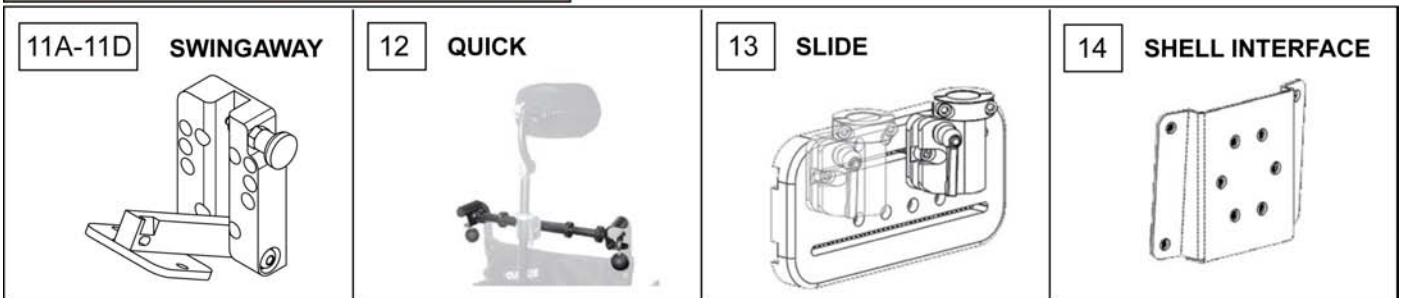


Pos.	Item Number	Description
1	M2500	AXYS DETACH BRKT
2	M2100CV	ONYX MOUNTING BRKT, DETACH
3	M2300	LINX DETACH MOUNT BRKT
4	M2200	PRO DETACH MOUNT BRKT
5	Create part number	COBRA DETACH MOUNT BRKT
6	M0030	KNOB PLAS ADJ LVR 1/4-20X20MM
7A	MW006030	COL SPLIT SHAFT 7/8" W/PIN BLK
7B	MW006032	COL .44TO.875 ADPTR W/PIN BLK
7C	MW006028	COL SPLIT SHFT 1-1/8" W/PIN BLK
8	MC100-WB	COVER 3/4" COLLAR/VINYL BLACK
9	M0055	FASTENER PACK DETACH
10	M0075	FASTENER PACK SWINGAWY MOUNT
11A	M5000	WHIT SWINGAWAY MOUNT BRKT
11B	M5100	ONYX MOUNTING BRKT, SWINGAWAY
11C	M5200	PRO MOUNTING BRKT, SWINGAWAY
11D	M5300	LINX MOUNTING BRKT, SWINGAWAY
12	M7000	ADJUSTABLE QUICK MOUNT
13	M7010	SLIDE MOUNT PLATE
14	M1010	PLAS SHELL IF PLATE

LATERAL FACIAL SUPPORT REPLACEMENT PARTS

BILATERAL

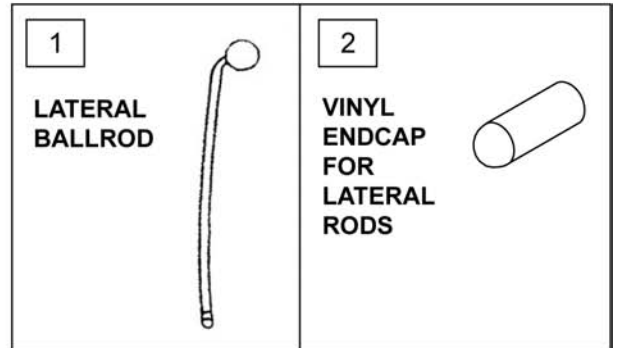


LATERAL FACIAL SUPPORT PARTS

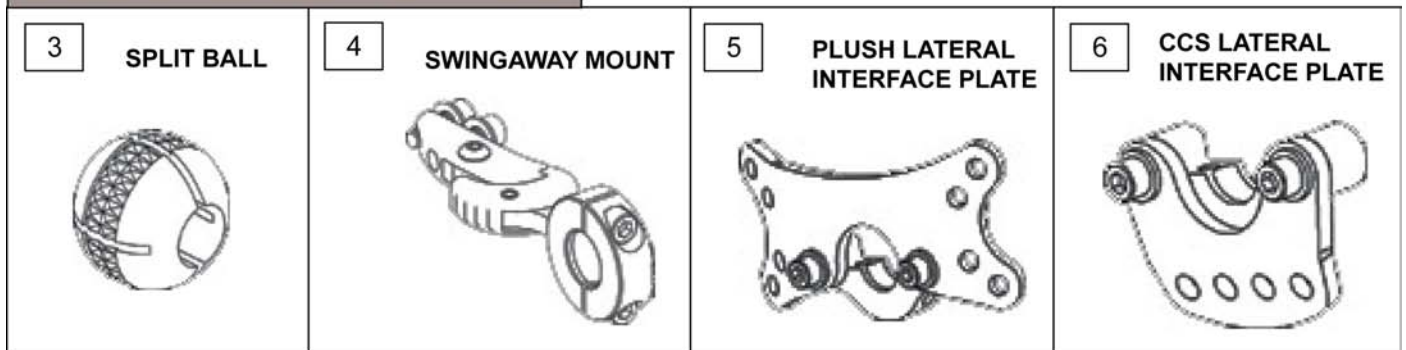


Pos.	Item Number	Description
1	LM1090	BALLROD, 3/4" CURVED LATERAL
2	MW000700	ENDCAP 5/16ID VINYL BLK
3	LM1100	3/4" SPLITBALL W/5/16"BORE SS
4	LM115	LFC MOUNT, SWINGAWAY
5	LM-PL	PLUSH LAT INTERFACE PLATE
6	LM-CCS	CONT CRADLE LAT INTERPLATE


ANTERIOR SUPPORTS DFS

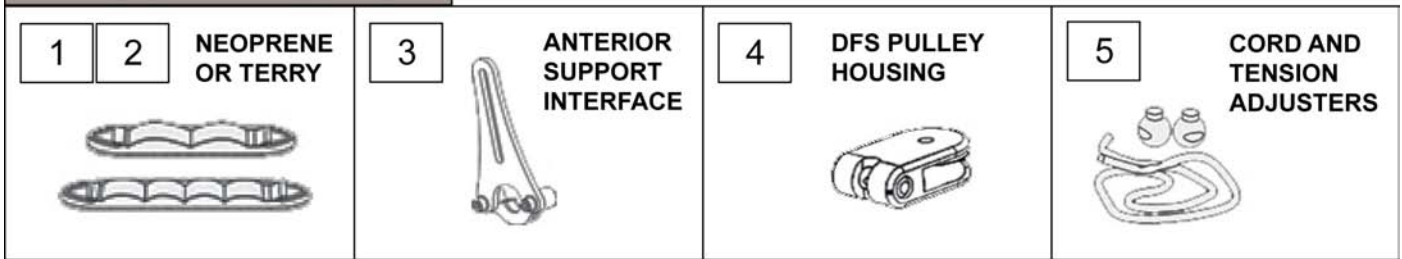

ANTERIOR SUPPORTS DFS2

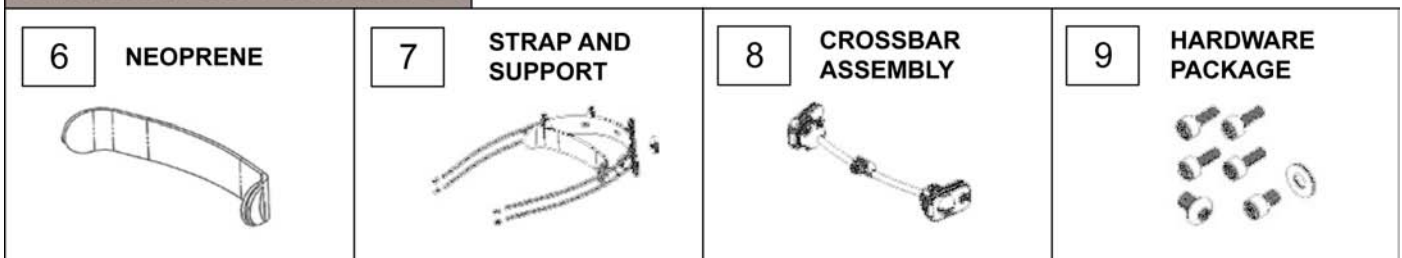

ANTERIOR AND LATERAL SUPPORTS INFANT

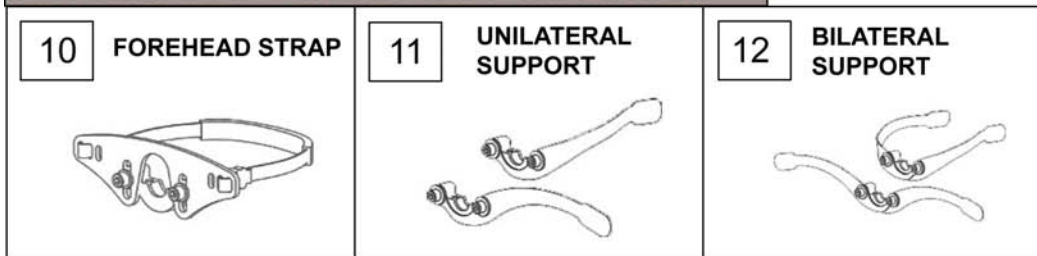


Pos.	Item Number	Description
1	D1017	DFS STRAP ADULT (TAS) BLK
1	D1028	DFS STRAP PEDIATRIC (TAS) BLK
2	D1018	DFS STRAP ADULT (NAS) BLK
2	D1027	DFS STRAP PEDIATRIC (NAS) BLK
3	D1010	ANT INTER PLATE, SINGLE PAD
4	D1015	DFS PULLEY HOUSING W/HRDWR
5	D1013	DFS CORD & 2 TENSION ADJ BALLS
6	D2018	NEOPRENE AGAINST SKIN ADULT
6	D2027	NEOPRENE AGAINST SKIN PED
7	DQ1010	STRAP AND SUPPORT ADULT
7	DQ3010	STRAP AND SUPPORT PED
8	DQ-CBA1	CROSSBAR ASSEMBLY ADULT
8	DQ-CBA3	CROSSBAR ASSEMBLY PED
9	DQ-HDW	HARDWARE PACKAGE
10	INFT500	INF STATIC ANTERIOR SUP STRAP
11	INFT700	PAD INFANT UNILAT BABY FLEECE
12	INFT750	PAD INF BILAT-BABY FLEECE CVR


ADULT AND PEDIATRIC

SINGLE SUB-OCCIPITAL


1

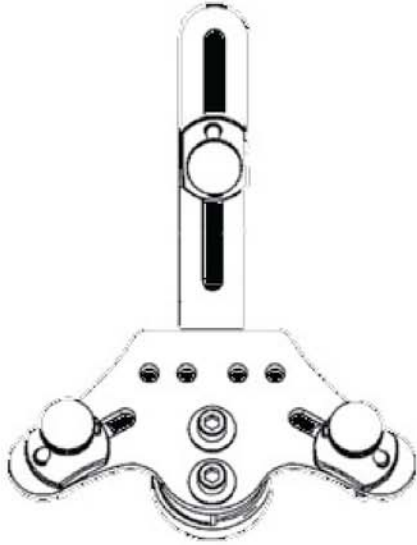

2

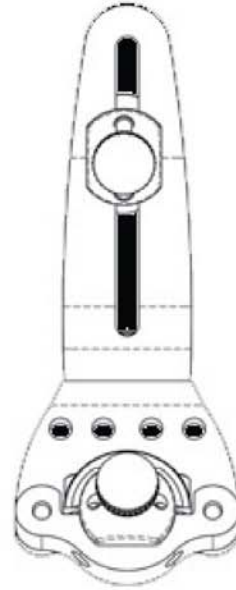


Pos.	Item Number	Description
1	M9375	S.O.F.T. BPA, DUAL SUB-OCC
2	M9275	S.O.F.T. BPA, SINGLE SUB-OCC

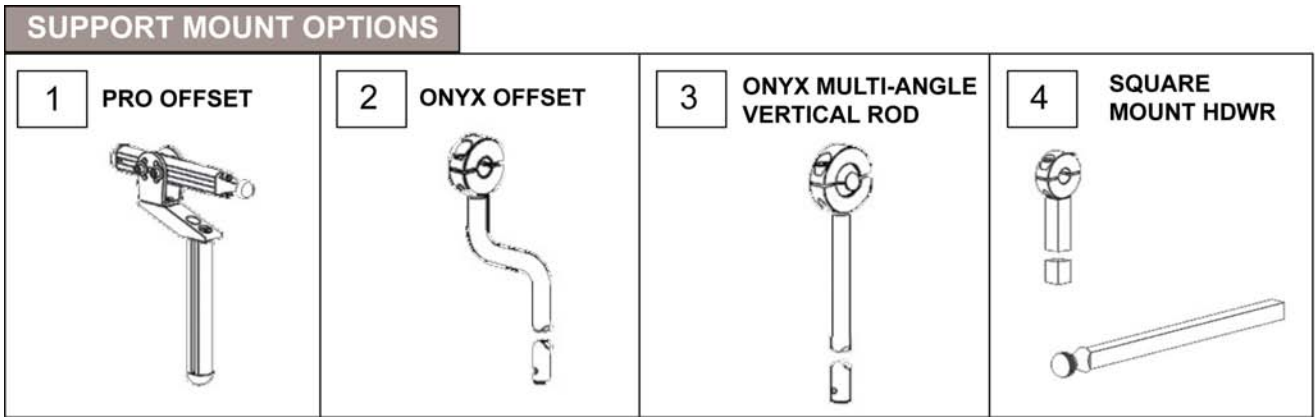


Pos.	Item Number	Description
1	PRC7070	PRO CORE MOUNT HDW, OFF-SET
2	OX6030	VERT ROD, 17.5" ONYX W/OFF-SET
3	OX6035	MULTI-ANG VERT ROD, 12" ONYX
3	OX6050	VERTICAL ROD 12"SQUARE ONYX
4	AX6055	AXYS 12" SQ HORIZ BAR BLK
4	AX5080	BALLROD AXYS 6" SQUARE BLACK

OPTIONS AND ACCESSORIES

ACCESSORIES

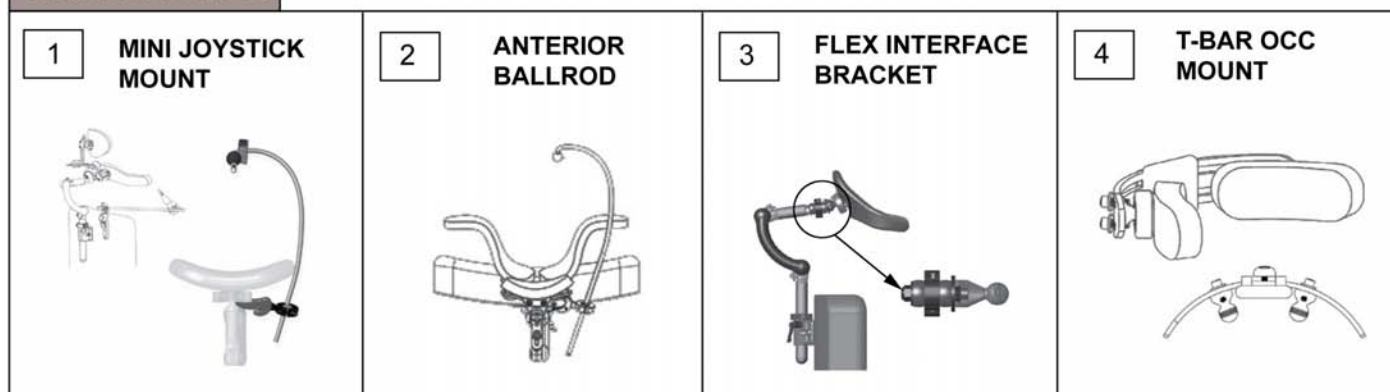

DURA-SOFT SWITCH

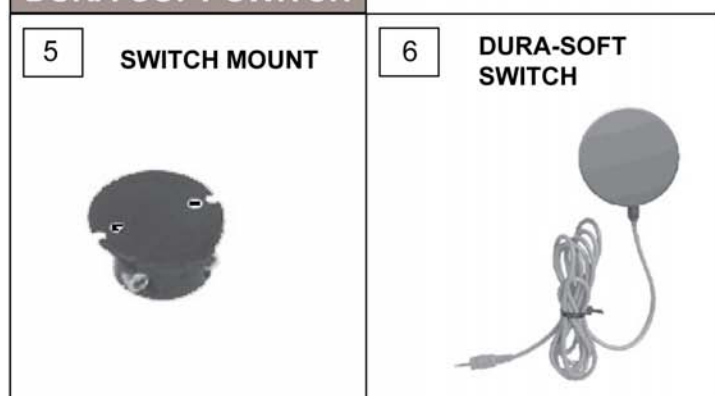

WHITMYER TOOL KIT



Pos.	Item Number	Description
1	LM1094	UNILAT MINI-JOYSTICK MT SYS
1	LM1092	UNILAT MIN-JOY MT W/PLUSH INTF
1	LM1093	UNILAT MINI-JOY MT W/CC INTERF
2	LM2094	ANTER BALLROD MT SYS FOR SOFT
2	LM2092	ANT BALLRD MT SY W/PLUSH INTF
2	LM2093	ANT BALLRD MT SYS W/CCS INTRFC
3	M9450	FLEX INTERFACE BRKT
4	PLT15L	OCCIPITAL T-BAR W/STND PADS,L
4	PLT15V	OCCIPITAL T-BAR W/STND PADS,R
4	PLT20L	OCCIPITAL T-BAR W/LONG PADS-L
4	PLT20V	OCCIPITAL T-BAR W/LONG PADS,R
5	FSO-8	SWITCHDURA S.O.F.T. W/3/4"MT
6	FSO-8A	DURA S.O.F.T. SWITCH
6	F700	PAD DUO 18X16
7	T2500	TOOL KIT, WHITMYER