INSTALLING THE HEADREST


The Headrest can be mounted to the backshell using 2x 2-hole slide nut, 2x M6x1x16 bolts, 2x 1/4-28 x 5/8" machine screws, and 4x washers.


I. High mount position Headrest installation 2

- a. Mount the headrest and bracket (A) as shown, using 2x M6 bolts and washers (B), and the 2-hole slide nut.
- b. After adjusting the headrest to optimum support position, use the 4mm hex key to tighten the screws


2. Low mount position Headrest installation 3 4 5


- a. To mount the headrest in the low position, invert the bracket(A).
- b. Using the 2x M6 bolts and washers (B), and the 2-hole slide nut, attach the bracket as shown.
- c. To orient the headrest (D) right-side up, use a 3/16" hex key to remove the two $1/4-28 \times 5/8$ headrest screws (C) orient the headrest as shown and re-attach the screws.
- d. After adjusting the headrest to optimum support position, use the 4mm and 3/16" hex keys to make sure the screws are tight

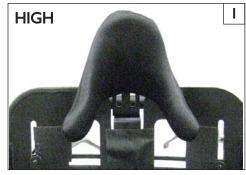

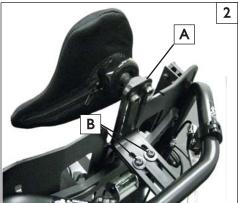



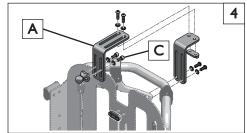

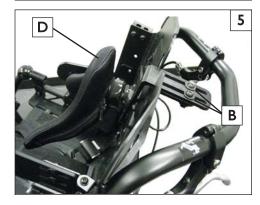

IS00131 Rev. A