


WARNING
Use the QTRONIX Programming Pad only
with suitable Penny & Giles
Programmable Controllers

Using your QTRONIX Programming Pad

- plug in the Programming Pad
- switch on the controller
- the Programming Pad is ready for use

Menus

- you are in a menu if the display shows a message ending with ?

- use the up and down keys to look through the menu



use the help key to find out what each menu item does



press the enter key to use a menu item

Once you are inside a menu item



use the help key to find out what to do next

Trying out your settings

- chair can operate with Programming Pad plugged in

When you finish

- unplug the QTRONIX Programming Pad
- the wheelchair is ready to drive