


# Wheel Lock Setup

- No-Go
- Setup Not Available
- \*Forward
- Rearward
- Short Rearward

## Long Attendant Wheel Lock POS

Rear Wheel Size	12	Roller Pos	Axle Pos						
			12	13	14	15	16	17	18
		1		2	2	1	1	1	1
		2		2	2	1	1	1	1
		3		3	3	2	2	1	1
		4			3	3	2	2	1
		5				3	3	2	2
		6					3	3	2
		16	Roller Pos	Axle Pos					
				12	13	14	15	16	17
	1		3	2	2	1	1	1	1
	2	1*	3	3	2	2	1	1	1
	3			3	3	2	2	1	1
	4				3	3	2	2	2
	5					3	3	2	2
	6						3	3	3
	20	Roller Pos	Axle Pos						
			12	13	14	15	16	17	18
	1	2*	2*	1*	3	3	2	2	2
	2	2*	2*	1*	1*	3	3	2	2
	3	3*	2*	2*	1*	1*	3	3	3
	4		3*	2*	2*	1*	1*	3	3
	5			3*	2*	2*	1*	1*	1*
	6				3*	2*	2*	1*	1*
	22	Roller Pos	Axle Pos						
			12	13	14	15	16	17	18
	1	2*	2*	1*	1*	3	3	2	2
	2	3*	2*	2*	1*	1*	3	3	3
	3	3*	3*	2*	2*	1*	1*	3	3
	4		3*	3*	2*	2*	1*	1*	1*
	5			3*	3*	2*	2*	1*	1*
	6				3*	3*	2*	2*	2*
	24	Roller Pos	Axle Pos						
			12	13	14	15	16	17	18
	1	3*	2*	2*	1*	1*	3	3	3
	2	3*	2*	2*	2*	1*	1*	3	3
	3		3*	3*	2*	2*	1*	1*	1*
	4			3*	3*	2*	2*	1*	1*
	5				3*	3*	2*	2*	2*
	6					3*	3*	2*	2*

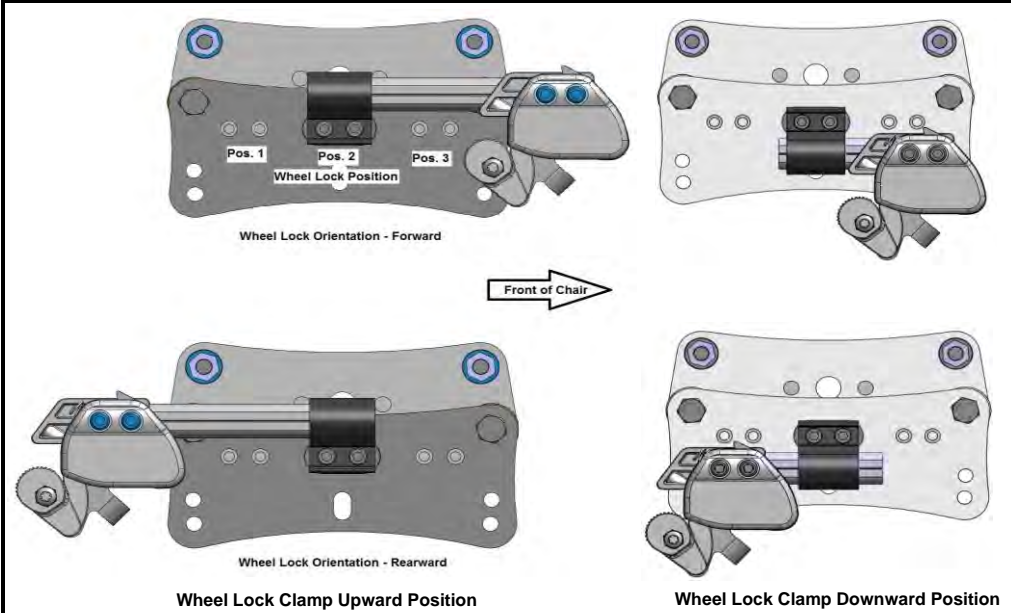

**Notes:**

- \* When Axle Plate Orientation is in 1-4 (ENG910), install wheel lock clamp in upward position
- \* When Axle Plate Orientation is in 5-8 (ENG910), install wheel lock clamp in downward position

# Wheel Lock Setup




Quickie IRIS

15-Jul-13

- No-Go
- Setup Not Available
- \*Forward
- Rearward
- Short Rearward

## Long Push to Lock Wheel Lock POS

Rear Wheel Size	Roller Pos	Axle Pos							
		12	13	14	15	16	17	18	
		12	1	2	2	2	1	1	1
2			3	2	2	2	1	1	
3	1*		1*	3	2	2	2	1	
4			1*	1*	3	2	2	2	
5				1*	1*	3	2	2	
6					1*	3	2	2	
16	1		1*	3	2	2	1	1	
	2	1*	1*	3	2	2	1		
	3	1*	1*	1*	3	2	1		
	4		1*	1*	1*	3	2		
	5			1*	1*	1*	3		
	6				2*	1*	1*		
	20	1	2*	2*	1*	3	3	2	
2		2*	2*	1*	1*	3	2		
3		3*	2*	2*	1*	1*	3		
4			3*	3*	2*	1*	1*		
5				3*	2*	2*	1*		
6					3*	2*	2*		
22		1	2*	2*	1*	1*	3	2	
	2	3*	2*	2*	1*	1*	3		
	3	3*	3*	2*	2*	1*	1*		
	4		3*	3*	2*	2*	1*		
	5			3*	3*	2*	2*		
	6				3*	3*	2*		
	24	1	3*	2*	2*	1*	1*	3	
2		3*	2*	2*	2*	1*	1*		
3		3*	3*	3*	2*	2*	1*		
4			3*	3*	3*	2*	2*		
5				3*	3*	2*	2*		
6					3*	3*	3*		

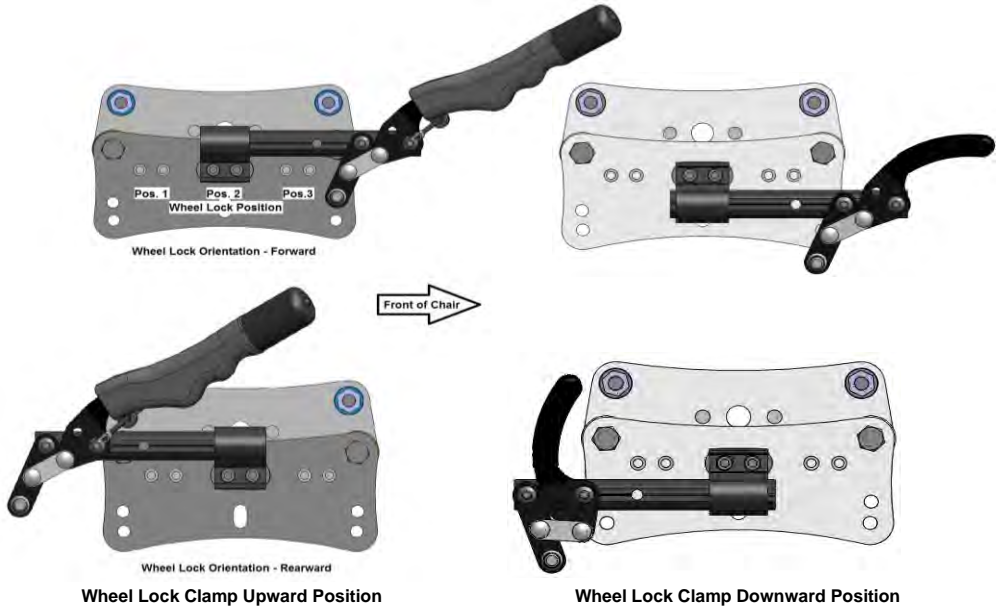

**Notes:**

- \* When Axle Plate Orientation is in 1-4 (ENG910), install wheel lock clamp in upward position
- \* When Axle Plate Orientation is in 5-8 (ENG910), install wheel lock clamp in downward position

# Wheel Lock Setup




Quickie IRIS

15-Jul-13

- No-Go
- Setup Not Available
- \*Forward
- Rearward
- Short Rearward

## Short Rearward Attendant Wheel Lock POS

Rear Wheel Size	12	Roller Pos	Axle Pos						
			12	13	14	15	16	17	18
		1	3						
		2	3						
		3							
		4							
		5							
6									
16	Roller Pos	Axle Pos							
		12	13	14	15	16	17	18	
	1	3							
	2								
	3								
	4								
	5								
6									
20	Roller Pos	Axle Pos							
		12	13	14	15	16	17	18	
	1								
	2								
	3								
	4								
	5								
6									
22	Roller Pos	Axle Pos							
		12	13	14	15	16	17	18	
	1								
	2								
	3								
	4								
	5								
6									
24	Roller Pos	Axle Pos							
		12	13	14	15	16	17	18	
	1								
	2								
	3								
	4								
	5								
6									

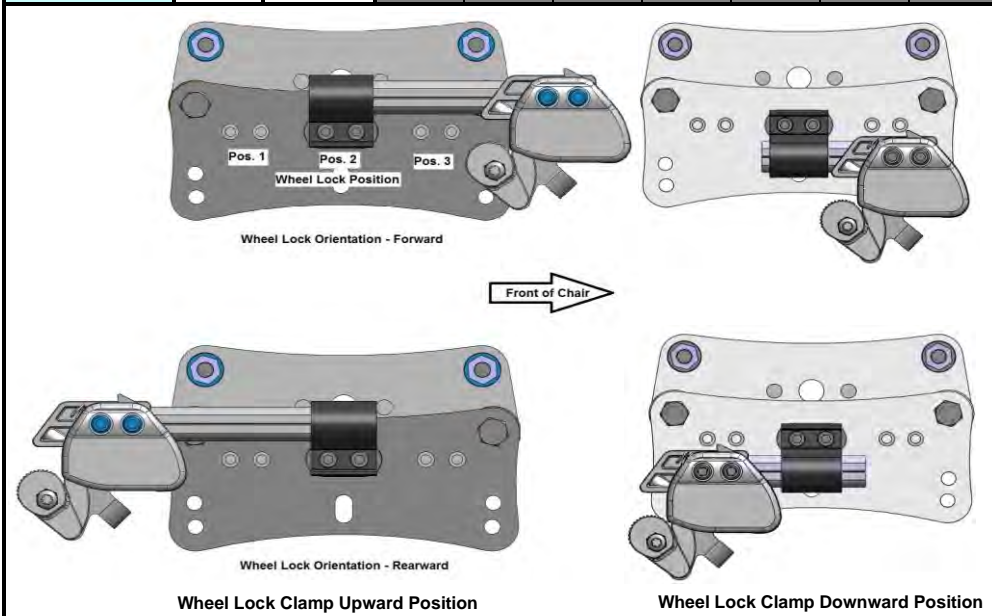

**Notes:**

- \* When Axle Plate Orientation is in 1-4 (ENG910), install wheel lock clamp in upward position
- \* When Axle Plate Orientation is in 5-8 (ENG910), install wheel lock clamp in downward position

# Wheel Lock Setup




Quickie IRIS

15-Jul-13

- No-Go
- Setup Not Available
- \*Forward
- Rearward
- Short Rearward

## Short Rearward Push to Lock Wheel Lock POS

Rear Wheel Size	12	Roller Pos	Axle Pos								
			12	13	14	15	16	17	18		
		1									
		2	3								
		3									
		4									
		5									
		6									
		16	Roller Pos	Axle Pos							
				12	13	14	15	16	17	18	
		1									
		2									
		3									
		4									
		5									
		6									
	20	Roller Pos	Axle Pos								
			12	13	14	15	16	17	18		
		1									
		2									
		3									
		4									
		5									
		6									
	22	Roller Pos	Axle Pos								
			12	13	14	15	16	17	18		
		1									
		2									
		3									
		4									
		5									
		6									
	24	Roller Pos	Axle Pos								
			12	13	14	15	16	17	18		
		1									
		2									
		3									
		4									
		5									
		6									

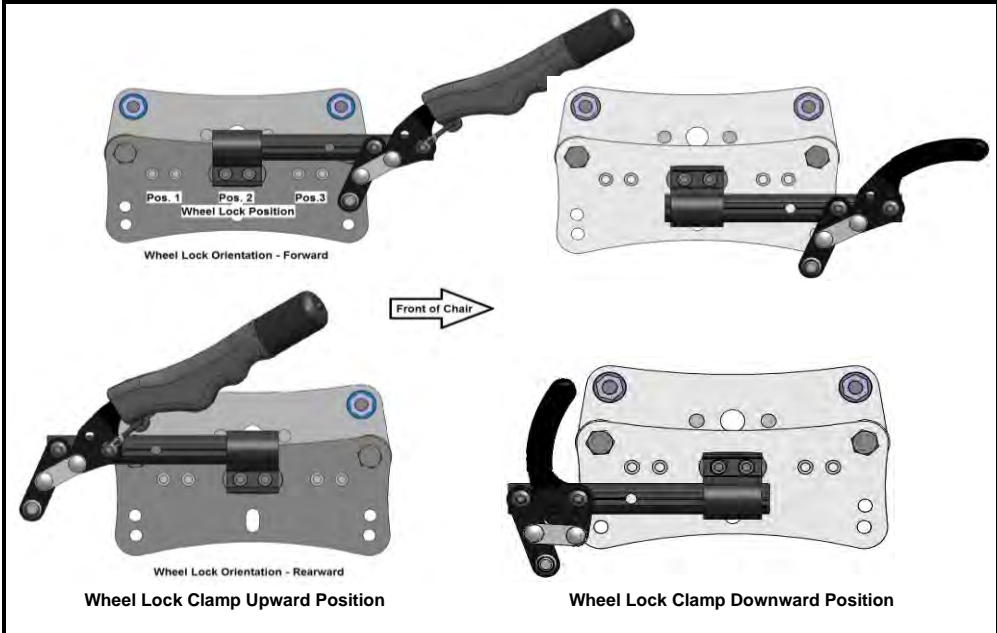

**Notes:**

- \* When Axle Plate Orientation is in 1-4 (ENG910), install wheel lock clamp in upward position
- \* When Axle Plate Orientation is in 5-8 (ENG910), install wheel lock clamp in downward position